Tube clamp adapters

DE Gebrauchsanweisung	4	CS Návod k použití.....	63
EN Instructions for use	8	RO Instrucțiuni de utilizare.....	68
FR Instructions d'utilisation.....	13	HR Upute za uporabu	72
IT Istruzioni per l'uso	18	SL Navodila za uporabo	77
ES Instrucciones de uso	22	SK Návod na používanie	81
PT Manual de utilização.....	27	BG Инструкция за употреба	86
NL Gebruiksaanwijzing.....	32	TR Kullanma talimatı	91
SV Bruksanvisning	36	EL Οδηγίες χρήσης	95
DA Brugsanvisning	41	RU Руководство по применению	100
NO Bruksanvisning	45	JA 取扱説明書.....	105
FI Käyttöohje.....	49	ZH 使用说明书.....	109
PL Instrukcja użytkowania.....	54	KO 사용 설명서.....	113
HU Használati utasítás	59		

1 Produktbeschreibung

Deutsch

INFORMATION

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INFORMATION

Date of last update: 2020-06-04

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

1.1 Construction and Function

Tube clamp adapters are used as components for modular lower limb prostheses. Together with the tube adapter they connect the prosthetic foot to the proximal components. Adapter combinations allow for controlled angle and length adaptation in the sagittal and frontal plane as well as adjustment of inward and outward rotation. These instructions for use apply to the following tube clamp adapters:

Diameter	Reference number
30 mm	4R21, 4R21=1, 4R52, 4R52=1, 4R56, 4R56=1, 4R56=2, 4R69, 4R69=AL
34 mm	4R82, 4R82=P, 4R91, 4R156, 4R156=1, 4R156=2, 4WR95=3

1.2 Combination possibilities

This prosthetic component is compatible with Ottobock's system of modular connectors. Functionality with components of other manufacturers that have compatible modular connectors has not been tested.

2 Intended use

2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

2.2 Area of application

Product approved for max. body weight

- The maximum approved body weight is specified in the technical data (see Page 12).

Additional constraints

- **4R21=1, 4R52=1:** Only for use in TF prostheses, below the prosthetic knee joint.
- **4R56*, 4R156*:** Only for use in hip disarticulation prostheses, above the prosthetic knee joint.

2.3 Environmental conditions

Storage and transport

Temperature range -20 °C to +60 °C (-4 °F to +140 °F), relative humidity 20 % to 90 %, no mechanical vibrations or impacts

4R21=1, 4R52, 4R52=1, 4R56, 4R56=1, 4R56=2, 4R69, 4R69=AL, 4R82, 4R82=P, 4R91

Allowable environmental conditions

Temperature range: -10 °C to +45 °C

Moisture: relative humidity: 20% to 90%, non-condensing

Unacceptable environmental conditions

Chemicals/liquids: fresh water, salt water, perspiration, urine, acids, soapsuds, chlorine water

Unacceptable environmental conditions

Solids: dust, sand, highly hygroscopic particles (e. g. talcum)

4R21, 4R156, 4R156=1, 4R156=2

Allowable environmental conditions

Temperature range: -10 °C to +45 °C

Moisture: relative humidity: 20% to 90%, non-condensing

Chemicals/liquids: fresh water as dripping water, occasional contact with salty air (e.g. near the ocean)

Solids: dust

Prohibited environmental conditions

Chemicals/moisture: salt water, perspiration, urine, acids, soapsuds, chlorine water

Solids: dust in high concentrations (e.g. construction site), sand, highly hygroscopic particles (e.g. talcum)

4WR95=3

Allowable environmental conditions

Temperature range: -10 °C to +45 °C

Chemicals/liquids: fresh water, soapsuds, chlorine water

Moisture: Submersion: max. 1 h in 2 m depth, relative humidity: no restrictions

Solids: Dust, occasional contact with sand

Clean the product after contact with humidity/chemicals/solids, in order to avoid increased wear and damage (see Page 11).

Prohibited environmental conditions

Solids: highly hygroscopic particles (e.g. talcum), dust in high concentrations (e.g. construction site), intensive contact with sand

Chemicals/liquids: salt water, perspiration, urine, acids, continuous use in liquid media

2.4 Service life

This product was tested by the manufacturer with 3 million load cycles. Depending on the user's activity level, this corresponds to a maximum service life of 5 years.

3 Safety

3.1 Explanation of warning symbols

CAUTION Warning regarding possible risks of accident or injury.

NOTICE Warning regarding possible technical damage.

3.2 General safety instructions

CAUTION
Excessive strain on the product
Risk of injury due to breakage of load-bearing components

- ▶ Use the product according to the specified area of application (see Page 9).

CAUTION
Unallowable combination of prosthetic components
Risk of injury due to breakage or deformation of the product

- ▶ Only combine the product with prosthetic components that are approved for that purpose.
- ▶ Based on the instructions for use of the prosthetic components, verify that they may be combined with each other.

CAUTION
Use under unallowable environmental conditions
Risk of injury due to damage to the product

- ▶ Do not expose the product to unallowable environmental conditions.

- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop etc.).

CAUTION
Exceeding the service life
Risk of injury due to change in or loss of functionality and damage to the product

- ▶ Ensure that the approved service life is not exceeded.

CAUTION
Mechanical damage to the product
Risk of injury due to change in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, do not continue using the product (see "Signs of changes in or loss of functionality during use" in this section).
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

Signs of changes in or loss of functionality during use
Among other factors, changes in functionality can be indicated by an altered gait pattern, a change in the positioning of the prosthetic components relative to each other and by the development of noises.

4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	-
1	Tube clamp adapter	-
For 4R21, 4R21=1, 4R52, 4R52=1, 4R56*		
4	Set screw	506G3=M8x12-V
For 4R69*, 4R82, 4R91, 4R156*, 4WR95=3		
4	Set screw	506G3=M8x14

Spare parts/accessories (not included in the scope of delivery)

Designation	Reference number
Single component pack for 4R156* Consisting of: Cap screw, clamping bolt, clamping bolt with interior thread, plastic ring	4D28

5 Preparing the product for use

⚠ CAUTION

Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- ▶ Observe the alignment and assembly instructions.

⚠ CAUTION

Improper assembly of the screw connections

Risk of injury due to breakage or loosening of the screw connections

- ▶ Clean the threads before every installation.
- ▶ Apply the specified torque values.
- ▶ Follow the instructions regarding the length of the screws and about how to secure the screws.

5.1 Mounting the adapter

⚠ CAUTION

Incorrect mounting of the tube

Risk of injury due to breakage of load-bearing components

- ▶ Slide the tube all the way to the stop in the intended prosthetic component when mounting.

> **Required materials:** torque wrench (e.g. 710D20), degreasing cleaner (e.g. acetone), 636K13 Loctite 241

- 1) Loosen the tube clamp cap screw by **two turns**.
- 2) **For final assembly:** Degrease the insertion range of the tube clamp adapter and tube adapter.
- 3) Insert the tube adapter fully into the tube clamp adapter up to the stop. Align the clamping slot:
Tube clamp adapter: **anterior**
Tube clamp adapter, sliding: **anterior** or **medial**
- 4) **4WR95=3 only:** Apply Loctite to the tube clamp cap screw.
- 5) Tighten the cap screw (tightening torque according to thread diameter: M4: **5 Nm**, M5: **10 Nm**, M6: **13 Nm**).

5.2 Installation in the modular prosthesis

> **Required materials:** torque wrench (e.g. 710D20), 636K13 Loctite 241

- ▶ Install the other prosthetic components as described in the respective instructions for use.

6 Cleaning

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

7 Maintenance

- ▶ A visual inspection and functional test of the prosthetic components should be performed after the first 30 days of use.

- ▶ Inspect the entire prosthesis for wear during normal consultations.
- ▶ Conduct annual safety inspections.

8 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

9.3 Warranty

The manufacturer warrants this device from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

10 Technical data

Tube diameter: 30 mm

Reference number	4R21	4R21=1	4R69	4R69=AL
Weight [g]	130	125	75	75
System height [mm]	33			
Material	Stainless steel		Aluminium	
Max. body weight [kg]	100	136	125	136

Reference number	4R52	4R52=1	4R56	4R56=1	4R56=2
Weight [g]	75	80	85	85	100
System height [mm]	33		34		35
Material	Titanium				
Angling °	-	-	10	20	30
Max. body weight [kg]	100	136	100		

Tube diameter: 34 mm

Reference number	4R82	4R82=P	4R91
Weight [g]	95	90	140
System height [mm]	33	-12	33
Material	Titanium		Stainless steel
Max. body weight [kg]	150		

Reference number	4R156	4R156=1	4R156=2	4WR95=3
Weight [g]	145	175	185	130
System height [mm]	36	37	38	33
Material	Titanium			

Reference number	4R156	4R156=1	4R156=2	4WR95=3
Angling [°]	10	20	30	–
Max. body weight [kg]	150			

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