**ottobock.**

## 50R153=\* Ottobock Fulcrum LSO

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## 50R153=\* Ottobock Fulcrum LSO

Thank you for using an Ottobock product. The following information provides information about the fit and use of the device. Please read the entire document carefully before use and ask questions for clarification.

Practitioner fitting instructions are provided on page 3.

### Indications for Use

- Post-operative stabilization protocol following spinal fusion, laminectomy/laminotomy, foraminotomy, laproscopic disk replacement, Intradiscal Electrothermal Therapy (IDET) procedure, multi-level decompression
- Burst Fracture
- Facet syndrome
- Chronic & Mechanical low back pain

### Contraindications for Use

Device should only be applied as instructed by a physician or practitioner. This device is not indicated for use with pregnancy, compromised circulation and/or pulmonary function, or any other cardiovascular or skeletal condition which could be compromised with the use of this device.

## User Instructions

### CAUTION

#### Usage Restrictions

This device is intended for single patient-use only. Reuse of the device is prohibited. The daily use of the device depends on the medical indication and the prescribing physician orders.

- Carefully follow all instructions provided by the prescribing physician and practitioner.

### How to Apply

1. Locate the handle on the left side of the device and fasten the handle near the pulley unit.
2. While standing in an upright position, hold the side panels apart, and center the back panel of the device along the spine.
3. Wrap the left side panel to the front to position the front panel. Then wrap the right side panel on top of the left side panel to close. Attach both sides in front. Adjust the front or back panels as needed.
4. Pull the handle out and away from the pulley unit to reach desired compression and support. Place the handle on the fabric fastener band securely.

### Tips

1. Be sure to start the handle near the pulley unit.
2. Extend or pull out the brace before application.
3. Adjust tension for sitting or standing to meet your needs.

### Cleaning and Care

### CAUTION

It is important to ensure the device is kept clean and dry. Do NOT use a washing machine, dryer, bleach, or harsh chemicals to clean the device.

1. Hand wash with mild soap and water.
2. Rinse thoroughly and towel or air dry.

**CAUTION**

Do not place the device near or on a heat source or open flame. DO NOT USE HEAT TO DRY THE DEVICE.

## Practitioner Fitting Instructions

### Measure for the Correct Size

1. Measure the circumference of the patient’s waist while standing.
2. Select appropriate size from chart below.

### Adult Sizing Guide

Device Size	Circumference	Posterior Panel Height	Anterior Panel Height	MODEL #
<b>Small</b>	25 – 30"	14"	7 ¼"	50R153=S
<b>Medium</b>	30 – 35"	14"	7 ¼"	50R153=M
<b>Large</b>	35 – 40"	15 ½"	7 ¼"	50R153=L
<b>XL</b>	40 – 45"	15 ½"	7 ¼"	50R153=XL
<b>XXL</b>	45 – 50"	15 ½"	9 ¼"	50R153=XXL
<b>XXXL</b>	50 – 55"	15 ½"	9 ¼"	50R153=XXXL

### Manufacturer’s Limited Warranty

Otto Bock HealthCare, LP (Ottobock) warrants all of its devices, to the original purchaser, to be free from defects in materials and workmanship. This warranty applies, subject to normal wear and tear, when the devices are used as intended, without unapproved modifications, following all Ottobock instructions and requirements; and when they are fitted by or under the direct supervision of certified/licensed practitioners. This Limited Warranty does not cover device damage caused by accidents, neglect, misuse or operation beyond capacity, parts damaged by improper installation, substitution of parts not approved by Ottobock, or any alteration or repair by others that, in Ottobock’s judgment, materially or adversely affect the device.

The duration of this Limited Warranty varies by product types and is effective from the date of delivery to the end-user. Please refer to [www.ottobockus.com](http://www.ottobockus.com) or call 800 328 4058 for questions. Ottobock’s sole obligation under this Limited Warranty shall be to repair, replace, refurbish the device at no charge, or refund the cost of the device to the original purchaser, at Ottobock’s sole discretion.

THE EXPRESS WARRANTIES SET FORTH ABOVE ARE IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING WITHOUT LIMITATION, ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, AND ALL SUCH OTHER WARRANTIES ARE HEREBY DISCLAIMED AND EXCLUDED BY OTTOBOCK. IN NO EVENT SHALL OTTOBOCK’S LIABILITY OF ANY KIND INCLUDE ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, EVEN IF OTTOBOCK SHALL HAVE BEEN ADVISED OF THE POSSIBILITY OF SUCH POTENTIAL LOSS OR DAMAGE.

Some states do not allow the exclusion of incidental or consequential damages, thereby rendering the aforementioned limitation in applicable to certain original purchasers.

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