C-Leg compact 3C96-1/ 3C86-1

DE Gebrauchsanweisung (Benutzer)	2
EN Instructions for use (user)	16
FR Instructions d'utilisation (Utilisateur)	29
ES Instrucciones de uso (Usuario)	43

INFORMATION

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INFORMATION

Date of last update: 12.05.2017

- Please read this document carefully before using the product.
- Have yourself instructed by qualified personnel in the proper and safe use of the product.
- Please contact the qualified personnel if you have questions about the product (e.g. regarding the start-up, use, maintenance, unexpected operating behaviour or circumstances). The contact information for the manufacturer is found on the back page.
- Please keep this document in a safe place.

The product „3C96-1/3C86-1 C-Leg compact“ is referred to as the product/prosthesis/knee joint/leg prosthesis/leg prosthesis solution below.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

The C-Leg compact is to be used **exclusively** for the exoprosthetic fitting of amputations of the lower limb.

1 Explanation of symbols

 **WARNING** Warnings regarding possible risks of severe accident or injury.

 **CAUTION** Warnings regarding possible risks of accident or injury.

2 Safety instructions

CAUTION

Non-observance of safety instructions. Failure to follow the below-mentioned safety instructions can lead to a faulty control or malfunction of the C-Leg compact and result in risk of injuries as well as damage to the C-Leg compact.

CAUTION

Manipulations on system components. Any changes or modifications you make to system components on your own initiative can lead to malfunction of the joint up to loss of function due to structural failure. This can cause you to fall.

- The opening and repairing of the joint may only be performed by authorized Ottobock technicians and the handling of the battery may only be carried out by Ottobock Service Centres (exchanges are not permissible). Your prosthetist is the contact person.

CAUTION

Risk of falling when climbing stairs. On stairs, always use the banister or handrail. When walking down stairs, the prosthetic foot should always be placed on the step so that the heel and posterior half of the foot contact the tread surface; the forefoot may extend beyond the riser to make knee flexion easier.

- Stop walking on stairs immediately whenever the sound warning beeps. Make careful tests to verify if the stance phase stabilization is active (see Section “Important User Instructions”).
- Pay attention to the vibration and sound warnings (beeps) of the C-Leg compact.
- Special caution is required when walking downstairs while carrying children.

CAUTION

Magnetic interferences. The joint can malfunction when near high-tension power lines, transmitters, transformers, CT scanners, or other sources of strong electromagnetic radiation (such as security systems for goods in department stores). This can cause you to fall.

Avoid proximity to strong magnetic and electric interference sources (e.g. transformer stations, high-powered radio or television transmitters).

CAUTION

Distance to HF communication devices is too small (e.g. mobile phones, Bluetooth devices, WiFi devices)

Falling due to unexpected behaviour of the product caused by interference with internal data communication.

Therefore, keeping the following minimum distances to these HF communication devices is recommended:

- Mobile phone GSM 850/GSM 900: 0.99 m
- Mobile phone GSM 1800/GSM 1900/UMTS: 0.7 m
- DECT cordless phones incl. base station: 0.35 m
- WiFi (routers, access points,...): 0.22 m
- Bluetooth devices (third-party products not approved by Ottobock): 0.22 m

CAUTION

Improper use of the joint. Any kind of overloading or excessive strain as well as improper use can lead to

- defects and result in malfunction of the joint;
- loss of function due to structural failure; or
- defects on the battery and hydraulic damper resulting in leakage of liquid.

This can cause you to fall as well as result in skin irritation.

- The C-Leg compact was designed for everyday activities and must not be used for unusual activities such as extreme sports (i.e. free climbing, paragliding, etc.).
- Careful handling of the prosthesis and its components not only increases their service life but, above all, ensures your personal safety! Should the prosthesis be subjected to unusual stresses (such as a fall), immediately contact your prosthetist and have the prosthesis inspected for any damage. If necessary, the responsible prosthetist will pass the prosthesis on to the Ottobock Service Centre.

WARNING

Risk of accident when driving a motor vehicle. The ability of leg prosthesis users to drive a motor vehicle is determined on a case-by-case basis. Criteria include the type of fitting (amputation level, unilateral or bilateral, residual limb conditions, design of the prosthesis) and the amputee's individual abilities. All persons are required to observe their country's national and state driving laws when operating motor vehicles. For insurance purposes, drivers should have their driving ability examined and approved by an authorized test centre. Ottobock recommends that the motor vehicle be professionally retrofitted to the user's individual needs (e.g. automatic shift). Risk-free driving must be ensured even when the prosthesis is not functioning.

⚠ CAUTION

Overheating of the hydraulic unit. Extended, continuous use (e.g. lengthy downhill walks) can lead to

- overheating of the hydraulic unit and result in malfunction of the joint; or
- defects on the hydraulic damper resulting in leakage of liquid.

This can cause you to fall as well as result in skin irritation. Touching overheated components can cause burns.

- Pay attention to the vibration signals that will occur in such cases to alert you that there is a risk of overheating. As soon as these vibrations begin, all activities must be stopped and the hydraulic unit be allowed to cool down. You may resume your activities once the vibration signals stop.
- If activities are continued despite the vibration signals, the hydraulic element may overheat and, in extreme cases, lead to a damage to the C-Leg compact. The joint should then be sent to an authorized Ottobock Service for inspection. Your prosthetist is the contact person.

⚠ CAUTION

Risk of falling when walking backwards. When putting down the toe first when walking backwards, the C-Leg compact can switch from the high stance phase resistance to swing phase resistance. Actively flexing the hip joint at this point in time can cause you to fall.

When walking backwards, secure the joint actively with the residual limb muscles.

⚠ CAUTION

Non-active safety mode. If the safety mode can no longer be activated, there is the risk that you will fall. If the C-Leg compact does not switch to the safety mode (e.g. due to short-circuit due to water penetration), you must actively stabilize the C-Leg compact at heel strike with your residual limb muscles until a prosthetist can be reached or a prosthesis replacement be accomplished.

⚠ CAUTION

Mechanical overloading. Exterior mechanical impact or stress such as shocks and vibrations can lead to

- short circuits in the electronics and battery and result in malfunction of the joint;
- defects on the battery and hydraulic damper resulting in leakage of liquid; or
- loss of function due to structural failure.

This can cause you to fall as well as result in skin irritation.

Do not expose system components to mechanical vibrations or shocks.

⚠ CAUTION

Penetration of dirt and humidity. Penetration of dirt and humidity into the system components can lead to

- short circuits in the electronics and battery and result in malfunction of the joint;
- defects on the hydraulic damper resulting in leakage of liquid; or
- loss of function due to structural failure.

This can cause you to fall as well as result in skin irritation.

- Do not let foreign particles or liquids enter the system components. Should the joint come into contact with liquid, remove the cosmetic cover and let the components dry. The joint must then be sent to an authorized Ottobock Service for inspection. Your prosthetist is the contact person.

- Always close the plastic cover of the protective elastomer cap after use.
- If the C-Leg compact comes into contact with salt water, immediately clean it with a cloth moistened with freshwater and let it dry. The joint must then be sent to an authorized Ottobock Service for inspection. Your prosthetist is the contact person.

CAUTION

Thermal overloading. Extended exposure to high temperatures can lead to defects and result in malfunction of the joint up to loss of function due to structural failure.

This can cause you to fall.

Avoid areas with extreme temperatures (see Section "Technical Information").

CAUTION

Self-discharge of the battery. When the joint is not in use for an extended period of time, self-discharge of the battery will result. This can lead to insufficient power supply to the electronics of the joint and result in undefined conditions. This can cause you to fall.

We therefore recommend recharging prior to every use.

CAUTION

Incorrect battery charging. Charging the battery with battery chargers that have not been approved by Ottobock can lead to defects and result in malfunction of the joint. This can cause you to fall.

Only use battery chargers approved by Ottobock for charging the battery of the C-Leg compact (also refer to Section "Charging the C-Leg compact prosthesis system").

CAUTION

Malfunction of the joint. Malfunctions of the joint can cause you to fall.

Pay attention to the vibration and sound warnings (beeps) of the C-Leg compact.

CAUTION

Risk of pinching where the joint bends. Ensure that fingers and other body parts are not in this area when bending the joint.

CAUTION

Improper use of the remote control. The remote control can get damaged by improper use. This can lead to malfunction of the remote control and result in unexpected actions of the joint. This can cause you to fall.

Observe the instructions for proper use of the remote control in Section "Remote Control".

INFORMATION

When using exoprosthetic knee joints, servomotoric, hydraulic, pneumatic or brake load dependent control functions can cause movement noise. This kind of noise is normal and unavoidable. Usually, it does not cause any problems. If the movement noise noticeably increases in the knee joint's life cycle, the joint should be inspected immediately by a prosthetist.

⚠ CAUTION

Results of product deterioration. Wear and tear on system components can lead to malfunction of the joint. This can cause you to fall.

For your own safety, comply with the specified service and inspection intervals (maintenance of operational safety and guaranty). Your prosthetist is the contact person.

INFORMATION

If you have been using forearm crutches or a cane during the C-Leg compact fitting, you will generally become more active within a short time. For that reason it is indispensable to check and possibly readjust the settings. A readjustment of the C-Leg compact settings by your prosthetist is also required if you switch from forearm crutches to a cane or to no walking aid at all.

3 Application

3.1 Intended use

The product is to be used **solely** for lower limb exoprosthetic fittings.

3.2 Area of application

Our components perform optimally when paired with appropriate components based upon weight and mobility grades identifiable by our MOBIS classification information and which have appropriate modular connectors.

The product is recommended for mobility grade 2 (restricted outdoor walker) and mobility grade 3 (unrestricted outdoor walker). Approved for a body weight of **up to 125 kg (275 lbs)**.

Exception: 2R80=110

The product is recommended for mobility grade 2 (restricted outdoor walker) and mobility grade 3 (unrestricted outdoor walker). Approved for a body weight of up to **100 kg (220 lbs)**.

3.3 Conditions of Use

The product was developed for everyday use and must not be used for unusual activities. These unusual activities include, for example, extreme sports (free climbing, parachuting, paragliding, etc.).

Permissible ambient conditions are described in the technical data (Technical data).

The product is intended exclusively for use on one user. Use of the product by another person is not approved by the manufacturer.

3.4 Indikation

For patients with knee disarticulation, transfemoral amputation and hip disarticulation

3.5 Qualification

The product may be fitted only by qualified personnel authorised by Ottobock after completing the corresponding training.

4 Functionality

This prosthesis is an electronic leg prosthesis system with hydraulic stance and swing phase control. The electronic system learns the individual phases of the step, allowing for a harmonious movement of the prosthetic lower limb. In other words, the C-Leg compact can adapt to different walking speeds and provides knee stability the moment it is needed. At heel contact you may notice the added hydraulic stance stability. Any unintentional bending of the knee such as may occur when walking on uneven terrain, etc. will be prevented. This knee joint makes it easy to comfortably walk downhill or go downstairs with a natural gait. Thanks to its microprocessor control, the prosthesis moves in a very natural way. During every phase of walking, the C-Leg compact offers you safety and comfort, regardless of your walking style, so you can move around carefree.

The C-Leg compact has two modes: The compact mode is intended for everyday use; in this mode, an additional standing function can optionally be activated and deactivated with the remote control.

The therapy mode enlarges the field of application of the C-Leg compact and offers additional functionality for early mobilization after amputation. This mode can only be adjusted by a prosthetist.

5 Application

5.1 Charging the C-Leg compact prosthesis system

CAUTION

Incorrect behaviour while charging the joint. If you walk while the battery charger is connected to the prosthesis, you can get caught on the cable and fall.

Take off the prosthesis prior to charging.

CAUTION

Incorrect handling while charging the joint. If the joint is not flexed completely during charging, the charging plug can get damaged and the joint can no longer be charged. This can lead to insufficient power supply to the electronics of the joint and result in undefined conditions. This can cause you to fall.

During charging, keep the C-Leg compact fully flexed.

INFORMATION

Prior to charging with the 4E50-* Battery Charger, have yourself instructed by your prosthetist using the 647G262 Instructions for Use.

INFORMATION

- The capacity of a fully charged battery is sufficient for one full day. We recommend charging the battery overnight when using the prosthesis on a daily basis.
- Charging is only possible at temperatures above 0 °C.

- 1 Bend the C-Leg compact until it hits the stop.
- 2 Open the plastic slide of the protective elastomer cap.
- 3 Connect the 4E50-* Charger and verify the charging status (see following figures).

Both LEDs are off.

No power supply
(or defective charger)

Both LEDs are lit.

Battery is being charged.
Battery capacity
is lower than 50%.

Yellow LED flashes;
green LED is lit.

Battery is being charged.
Battery capacity
is above 50%.

Yellow LED is off;
green LED is lit.

Battery is fully charged
(or connection with C-Leg
compact is interrupted).

- 4 After charging, remove the charging plug (C-Leg compact performs self-test) and close the plastic slide before you extend the prosthesis.

6 Remote control

The C-Leg compact has two modes: **The compact mode** is intended for everyday use; in this mode, an additional standing function can optionally be activated and deactivated with the remote control.

The **therapy mode** enlarges the field of application of the C-Leg compact and offers additional functionality for early mobilization after amputation. This mode allows to permanently lock the knee joint in full extension. By pressing a button on the remote control, the joint can be unlocked, e.g. for sitting down.

INFORMATION

The mode can only be changed by a prosthetist with the C-Soft adjustment software.

⚠ CAUTION

Improper use of the remote control. When using the remote control, the functionality of the joint changes. In certain situations, this can cause you to fall.

When using the remote control, you must always stand securely. Then carefully try, whether the desired function has been activated.

⚠ CAUTION

Penetration of water in the remote control. The remote control is not waterproof. If water penetrates the remote control, the device may become damaged (guaranty will become void). This can lead to malfunction of the remote control, result in unexpected actions of the joint, and cause you to fall. Should water have penetrated the remote control, the device should be dried at room temperature for at least one day. Before starting to reuse the remote control, return it to an authorized Ottobock Service for inspection. Your prosthetist is the contact person.

⚠ CAUTION

Manipulation on the remote control. Any changes or modifications you make to the remote control on your own initiative can lead to malfunction and result in unexpected actions of the joint. This can cause you to fall.

Any changes or modifications to the device may limit its use.

INFORMATION

Important user instructions on the use of the remote control. Keep the C-Leg compact and the residual limb still (no bending or extending!).

For safety purposes, the working range of the remote control is limited to approximately 70 cm. However, the remote control will only function properly if held at least 30 cm away from the joint. If the activation/deactivation per remote control was not performed, ensure that the remote control is placed between 30 cm and 70 cm of the C-Leg compact and repeat the command. If you are still not successful, perform a pairing process (see Section "Pairing").

7 Modes and use of the remote control

7.1 Compact mode:

Upon delivery the knee joint is in the compact mode. If required, the joint can be locked or unlocked in the flexion direction at a flexion angle between 0° and 30° (standing function) with the remote control. For safety reasons, movement in the extension direction is always possible; in full extension position the joint remains locked (locking function)

7.1.1 To activate the standing function (Fig. 1):

Press the remote control button for 2 seconds (the joint emits 2 beep sounds at intervals of one second). Two long beep sounds and two vibration signals follow to confirm the activation.

INFORMATION Operating the remote control at a flexion angle larger than 30° exclusively activates the locking function: **The joint automatically locks once full extension is reached.**

7.1.2 To deactivate the standing- or locking function:

Again press the remote control button for 2 seconds. One long beep sound and one vibration signal follow to confirm unlocking.

INFORMATION The standing function can only be activated and deactivated, when the knee joint is kept still during operation of the remote control.

INFORMATION If the standing or locking function has been activated and the remote control is defective or the battery is empty, the knee joint can be unlocked by connecting the battery charger to the C-Leg compact. This works only in the compact mode.

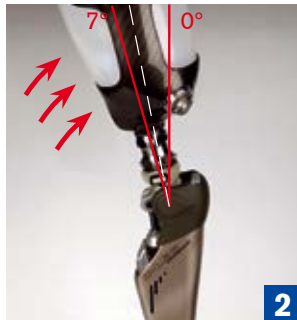
7.1.3 Therapy mode:

The therapy mode can only be activated by the prosthetist with the C-Soft adjustment software (from version 2.0). Once activated, the joint is locked in full extension, for example for therapeutic applications for early mobilization after amputation. If required, the joint can be unlocked with the remote control (e.g. for sitting down). The hydraulic damping of the system supports the prosthesis wearer when sitting down or standing up. The prosthesis can be loaded.

7.1.4 To unlock the knee joint:

Press the remote control button for 2 seconds (the joint emits 2 beep sounds at intervals of one second). One long beep sound and one vibration signal follow to confirm unlocking. Now the knee joint is unlocked and can be flexed, e.g. for sitting down (Fig. 1). The joint remains in stance phase damping until it is brought into extension again.

As soon as the knee angle during extension is less than 7°, the joint automatically activates the lock in the flexion direction. This is confirmed by 2 long beep sounds and 2 vibration signals (Fig. 2).

INFORMATION If the knee joint is not flexed more than 7° within 7 seconds after unlocking, it automatically locks again for safety reasons. During this period, the remote control cannot be used.

7.1.5 Attachment of the Remote Control

The remote control can be:

- 1 attached to a key lanyard (Fig. 1);
- 2 attached to the belt of the prosthesis wearer with a clip (Fig. 2); or
- 3 attached to a holder provided on the socket (integrated in the laminate, attached with buckles or glued in place; Fig. 3).

Pairing (Establishing the Communication between the C-Leg compact and Remote Control)

Pairing does not have to be performed upon delivery. Pairing must be done

- as soon as a new remote control (replacement) is used with the C-Leg compact.
- if a switch cannot be effected with fully charged batteries (of both the joint and the remote control) within the working range of the remote control (30–70 cm).
- upon receipt of the remote control with a new battery.

7.1.6 Execution:

Briefly press the start button by poking a thin object (e.g. a paper clip) through the small hole of the remote control. The joint will confirm the successful pairing with 5 short beep signals.

CAUTION **Fault while connecting the C-Leg compact and remote control (pairing).** If several joints are present in close proximity, this can result in unwanted connection of the remote control with another joint (pairing). This can cause you to fall. To ensure identification of the remote control with the corresponding joint, no other joint may be present within a radius of 3 m during the pairing process.

7.1.7 Changing the Battery of the Remote Control

CAUTION

Impermissible replacement of the battery of the remote control. Replacing the battery on your own initiative can result in defects of the remote control. Malfunction and subsequent unexpected actions of the joint are then possible. This can cause you to fall.

The battery of the remote control may only be changed by an authorized Ottobock Service. The battery is automatically replaced at the biennial service intervals. Therefore, please always return the remote control along with the C-Leg compact.

To change a battery outside of the biennial service intervals, send the remote control to an authorized Ottobock Service. Upon receipt of the remote control with a new battery, the remote control must be paired with the C-Leg compact. Your prosthetist is the contact person.

8 Important User Instructions

8.1 Safety mode

Aside from the normal operation mode, the C-Leg compact has a safety mode. As soon as a critical error occurs in the system or when the battery is almost empty, the C-Leg compact automatically switches to the safety mode. In that mode, the prosthesis activates a high flexion damping (high safety/reduced comfort), which makes it possible to walk safely despite a non-active system. The switch to the safety mode is announced immediately prior to the switch with sound and vibration signals. If the safety mode was prompted by an empty battery, recharge the battery. The joint will then switch back automatically to the operation mode.

⚠ CAUTION Non-active safety mode. If the safety mode can no longer be activated, there is the risk that you will fall. If the C-Leg compact does not switch to the safety mode (e.g. due to short-circuit due to water penetration), you must actively stabilize the C-Leg compact at heel strike with your residual limb muscles until a prosthetist can be reached or a prosthesis replacement be accomplished.

⚠ CAUTION Danger when activating the safety mode. Upon activation of the safety mode, the damping behaviour of the joint changes. In certain situations, this can cause you to fall. As soon as the sound and vibration signals go off simultaneously, you must stop all activities with the leg prosthesis. After approximately 10 seconds, and from a secure standing position, check to see if the safety mode with the high flexion damping has been activated by slightly bending the C-Leg compact repeatedly under controlled weight bearing.

⚠ CAUTION Safety mode cannot be deactivated. If the joint is exclusively in the safety mode, the joint has a defect. When in this condition, malfunctions can occur. This can cause you to fall. If the safety mode could not be deactivated by recharging the battery, an error has occurred. Contact your prosthetist for troubleshooting.

8.2 Safety signal for incorrect connection to the tube adapter

If the tube adapter is somehow disconnected from the C-Leg compact joint while in operation and with sufficient battery capacity, the C-Leg compact will emit short beeps and, for approximately five minutes, vibrate with slow, pulsing signals. The C-Leg compact will activate the safety mode and maintain it until the tube adapter cable is correctly attached (provided the battery capacity remains sufficient).

⚠ CAUTION Safety signal occurs. The safety signal indicates an error of the joint.

When in this condition, malfunctions can occur. This can cause you to fall. In the event such a safety signal occurs, stop all activities with the C-Leg compact and have the error resolved by your prosthetist.

8.3 Battery capacity

During normal operation, the battery capacity is indicated with different vibration signals:

Battery capacity	Vibration impulse
Approx. 1 hour of operation remains	3 ×
Approx. 30 minutes of operation remains	5 ×
Immediately prior to switching into the safety mode	10 ×

8.4 Ventilation

Air may accumulate in the hydraulic unit if the C-Leg compact is not stored in an upright position. This is noticeable through sounds and irregular damping behaviour.

The automatic ventilation mechanism ensures that all functions of the C-Leg compact are again intact after approximately 10 – 20 steps.

8.5 Cleaning

- Clean the product with a damp cloth and mild soap (e.g. Ottobock 453H10=1 Derma Clean) when needed.
Ensure that no liquid penetrates into the system component(s).
- Dry the product with a lint-free cloth and allow it to air dry fully.

9 Completing information

9.1 Service intervals

To ensure proper function and for your own safety, a maintenance service is required every two years. This service includes the inspection of the sensors and the replacement of worn parts.

To obtain the service, send in the C-Leg compact with mounted tube adapter as well as the complete charger unit including the AC adapter.

INFORMATION

This component was tested for three million load cycles in accordance with ISO 10328. Depending on the patient's activity level, this corresponds to a service life of three to five years.

The duration of use can be individually extended depending on the intensity of use by making use of regular service inspections.

9.2 Technical data

- max. flexion angle 135°
- Operating voltage of the battery charger: 100 V to 260 V AC
- Charging time: approx. 3.5 h
- Charging cycles (charging and discharging cycles) after which at least 80% of the original battery capacity remains available: 500
- Operating frequency of the battery charger: 50 Hz to 60 Hz
- Storage and transport in original packaging: -10 °C/+14 °F to +60 °C/+140 °F
- Storage and transport without packaging: -10 °C/+14 °F to +60 °C/+140 °F
max. 80% relative humidity, non-condensing
- Operation: -10 °C/+14 °F to +60 °C/+140 °F
max. 80% relative humidity, non-condensing
- Charging the battery: 0 °C/+32 °F to +50 °C/+122 °F
- Battery type: Li-ion
- Operating time of prosthesis with fully charged battery: 1 day with average use
- Product behaviour during charging: the product is non-functional
- Range of the remote control: 30–70 cm
- Service life of the remote control battery: 2 years
- Weight of the prosthesis without tube adapter: approx. 1210 g
- Weight of the tube adapter: 2R80: approx. 178 g to 256 g
2R81: approx. 439 g to 482 g

9.3 Symbols Used

Declaration of conformity according to the applicable European directives

Legal manufacturer

This product must not be disposed of with regular domestic waste. Disposal that does not comply with the regulations of your country may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in your country regarding return and collection processes.

Serial number of the device

Caution, hot surface

Compliance with the requirements according to „FCC Part 15“ (USA)

Compliance with the requirements according to the „Radiocommunications Act“ (AUS)

Non-ionising radiation

Lot number

10 Liability

The device is only to be used under the specified conditions and for the intended purposes. The device must be maintained according to the Instructions for Use. The device must only be operated with tested modular components in accordance with the Ottobock Mobility System. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

The C-Leg compact prosthesis system is designed exclusively for use on one patient. Use of the product by other persons is not approved by the manufacturer.

11 CE conformity

The electronic 3C96-1/3C86-1 C-Leg compact Knee Joint System meets the requirements of the 93/42/EEC guidelines for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by Ottobock with sole responsibility according to appendix VII of the guidelines. The electronic 3C96-1/3C86-1 C-Leg compact Knee Joint System also meets the requirements of the 1999/5/EC guidelines for radio equipment and telecommunications terminal equipment. Conformity assessment was carried out by Ottobock according to appendix IV of the guidelines. A copy of the declaration of conformity can be requested from the manufacturer (see back side).

The product meets the requirements under the RoHS Directive 2011/65/EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment.

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A series of 20 horizontal lines spaced evenly down the page, providing a template for handwriting practice.

Kundenservice/Customer Service

Europe

Otto Bock HealthCare
Deutschland GmbH
37115 Duderstadt · Germany
T +49 (0) 5527 848-3411
F +49 (0) 5527 848-1414
healthcare@ottobock.de

Otto Bock Healthcare
Products GmbH
1070 Wien · Austria
T +43 (0) 1 5269548
F +43 (0) 1 5267985
vertrieb.austria@ottobock.com

Otto Bock
Adria Sarajevo D.O.O.
71000 Sarajevo
Bosnia-Herzegovina
T +387 (0) 33 766200
F +387 (0) 33 766201
obadria@bih.net.ba

Otto Bock Bulgaria Ltd.
1612 Sofia · Bulgaria
T +359 (0) 2 80 57 980
F +359 (0) 2 80 57 982
info@ottobock.bg

Otto Bock Suisse AG
CH-6036 Dierikon
T +41 (0) 41 455 61 71
F +41 (0) 41 455 61 70
suisse@ottobock.com

Otto Bock ČR s.r.o.
33008 Zruč-Senec
Czech Republic
T +420 (0) 377825044
F +420 (0) 377825036
email@ottobock.cz

Otto Bock Iberica S.A.
28760 Tres Cantos (Madrid)
Spain
T +34 (0) 91 8063000
F +34 (0) 91 8060415
info@ottobock.es

Otto Bock France SNC
91978 Courtaboeuf Cedex
France
T +33 (0) 1 69188830
F +33 (0) 1 69071802
information@ottobock.fr

Otto Bock Healthcare plc
Egham, Surrey TW20 0LD
United Kingdom
T +44 (0) 1784 744900
F +44 (0) 1784 744901
bockuk@ottobock.com

Otto Bock Hungária Kft.
1135 Budapest · Hungary
T +36 (0) 1 4511020
F +36 (0) 1 4511021
info@ottobock.hu

Otto Bock Adria d.o.o.
10431 Sveta Nedelja · Croatia
T +385 (0) 1 3361 544
F +385 (0) 1 3365 986
ottobockadria@ottobock.hr

Otto Bock Italia Srl Us
40054 Budrio (BO) · Italy
T +39 (0) 051 692-4711
F +39 (0) 051 692-4710
info.italia@ottobock.com

Otto Bock Benelux B.V.
5692 AK Son en Breugel
The Netherlands
T +31 (0) 499 474585
F +31 (0) 499 476250
info.benelux@ottobock.com

Industria Ortopédica
Otto Bock Unip. Lda.
1050-161 Lisboa · Portugal
T +351 (0) 21 3535587
F +351 (0) 21 3535590
ottobockportugal@
mail.telepac.pt

Otto Bock Polska Sp. z o. o.
61-029 Poznań · Poland
T +48 (0) 61 6538250
F +48 (0) 61 6538031
ottobock@ottobock.pl

Otto Bock Romania srl
077405 Chitila, Jud. Ilfov
Romania
T +40 (0) 21 4363110
F +40 (0) 21 4363023
info@ottobock.ro

OOO Otto Bock Service
143441 Moscow Region/
Krasnogorskiy Rayon
Russian Federation
T +7 (0) 495 564 8360
F +7 (0) 495 564 8363
info@ottobock.ru

Otto Bock Scandinavia AB
60114 Norrköping · Sweden
T +46 (0) 11 280600
F +46 (0) 11 312005
info@ottobock.se

Otto Bock Slovakia s.r.o.
851 01 Bratislava 5
Slovak Republic
T +421 (0) 2 32 78 20 70
F +421 (0) 2 32 78 20 89
info@ottobock.sk

Otto Bock Sava d.o.o.
18000 Niš · Republika Srbija
T +381 (0) 18 4285888
F +381 (0) 18 4539191
info@ottobock.rs

Otto Bock Ortopedi ve
Rehabilitasyon Tekniği Ltd. Şti.
34387 Mecidiyeköy-Istanbul
Turkey
T +90 (0) 212 3565040
F +90 (0) 212 3566688
info@ottobock.com.tr

Africa

Otto Bock Algérie E.U.R.L.
Mackle-Ben Aknoun
Alger · DZ Algérie
T +213 (0) 21 913863
F +213 (0) 21 913863
information@ottobock.fr

Otto Bock Egypt S.A.E.
Mohandessein · Giza · Egypt
T +202 (0) 330 24 390
F +202 (0) 330 24 380
info@ottobock.com.eg

Otto Bock South Africa (Pty) Ltd
Johannesburg · South Africa
T +27 (0) 11 312 1255
info-southafrica@ottobock.co.za

Americas

Otto Bock Argentina S.A.
CP 1426 Ciudad Autónoma de
Buenos Aires · Argentina
T +54 (0) 11 4706-2255
F +54 (0) 11 4788-3006
atencionclientes@
ottobock.com.ar

Otto Bock do Brasil Ltda.
13051-030 Campinas-
São Paulo · Brasil
T +55 (0) 19 3729 3500
F +55 (0) 19 3269 6061
ottobock@ottobock.com.br

Otto Bock HealthCare Canada
Burlington, Ontario, L7L 5N5
Canada
T +1 (0) 289 288-4848
F +1 (0) 289 288-4837
infocanada@ottobock.com

Otto Bock HealthCare
Andina Ltda.
Bogotá · Colombia
T +57 (0) 1 8619988
F +57 (0) 1 8619977
info@ottobock.com.co

Otto Bock de Mexico
S.A. de C.V.
C.P. 01180 México, D.F.
Mexico
T +52 (0) 55 5575 0290
F +52 (0) 55 5575 0234
info@ottobock.com.mx

Otto Bock HealthCare
Minneapolis, MN 55447 · USA
T +1 (0) 763 553 9464
F +1 (0) 763 519 6153
usa.customerservice@
ottobockus.com

Asia/Pacific

Otto Bock Australia Pty. Ltd.
Baulkham Hills NSW 2153
Australia
T +61 (0) 2 8818 2800
F +61 (0) 2 8814 4500
healthcare@ottobock.com.au

Beijing Otto Bock Orthopaedic
Industries Co., Ltd.
Beijing, 100015, P.R. China
T +8610 (0) 8598 6880
F +8610 (0) 8598 0040
news-service@ottobock.com.cn
www.ottobock.com.cn

Otto Bock Asia Pacific Ltd.
Wanchai, Hong Kong · China
T +852 (0) 2598 9772
F +852 (0) 2598 7886
info@ottobock.com.hk

Otto Bock HealthCare India
Mumbai, 400071 · India
T +91 (0) 22 2520 1268
F +91 (0) 22 2520 1267
information@indiaottobock.com

Otto Bock Japan K. K.
Tokyo, 108-0023 · Japan
T +81 (0) 3 3798-2111
F +81 (0) 3 3798-2112
ottobock@ottobock.co.jp

Otto Bock Korea
HealthCare Inc.
137-070 Seoul · Korea
T +82 (0) 2 577-3831
F +82 (0) 2 577-3828
info@ottobockkorea.com

Otto Bock
South East Asia Co., Ltd.
Bangkok 10900 · Thailand
T +66 (0) 2 930 3030
F +66 (0) 2 930 3311
obsea@ottobock.co.th

Legal information that applies exclusively to specific countries is written in the official language of the respective country of use after this chapter.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- 1) This device may not cause harmful interference, and
- 2) This device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/ TV technician for help.

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Caution: Exposure to Radio Frequency Radiation.

This device must not be co-located or operating in conjunction with any other antenna or transmitter.

Responsible party:

Otto Bock Health Care, LP
3820 West Great Lakes Drive
Salt Lake City, Utah 84120-7205 USA
Phone + 1-801-956-2400
Fax + 1-801-956-2401

This device complies with RSS 210 of Industry Canada.

Operation is subject to the following two conditions:

- (1) this device may not cause interference, and
- (2) this device must accept any interference, including interference that may cause undesired operation of this device.

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Caution: Exposure to Radio Frequency Radiation.

The installer of this radio equipment must ensure that the antenna is located or pointed such that it does not emit RF field in excess of Health Canada limits for the general population; consult Safety Code 6, obtainable from Health Canada's website <http://www.hc-sc.gc.ca/rpb>.

Responsible party:

Otto Bock Healthcare Canada Ltd.
5470 Harvester Road
L7L 5N5 Burlington, Ontario
Canada
Phone + 1-800-665-3327

Caution: Federal law (USA) restricts this device to sale by or on the order of a practitioner licensed by law of the State in which he/she practices to use or order the use of the device.

Otto Bock Healthcare Products GmbH
Brehmstraße 16 · 1110 Vienna · Austria
T +43-1 523 37 86 · F +43-1 523 22 64
info.austria@ottobock.com · www.ottobock.com

Ottobock has a certified Quality Management System in accordance with ISO 13485.