FIT KIT for WalkOn Flex

Quality for life

FIT KIT for WalkOn Flex

DE	Gebrauchshinweise	4
EN	Instruction folder	6
IT	Vedere allegato	8
ES	Carpeta de instrucciones	10
NL	Instructiefolder	12
NO	Bruksanvisning	14
SV	Instruktionsfolder	16
DA	Brugsanvisning	18
CS	Pokyny pro použití	20
FI	Ohjekansio	22

Best.-Nr. 28T2

WalkOn FIT KIT für 28U22 WalkOn Flex

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Art. no. 28T2

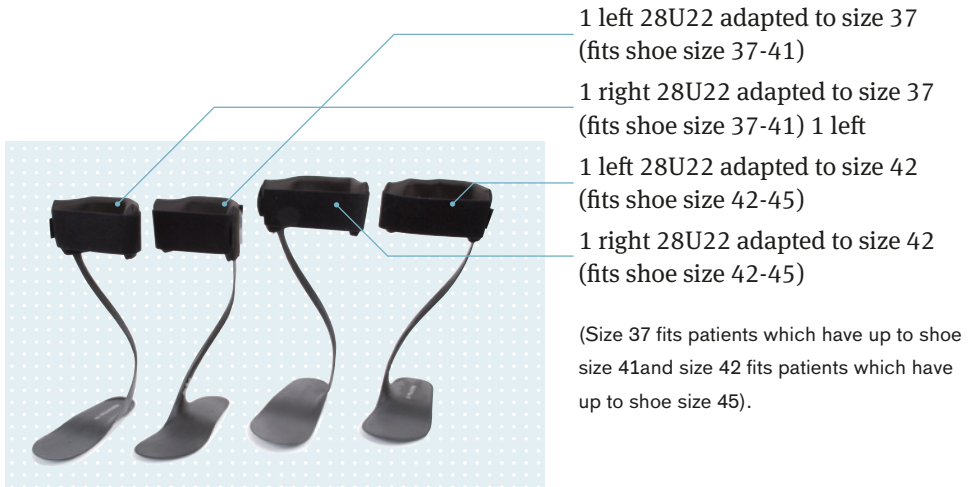
WalkOn FIT KIT for 28U22 WalkOn Flex

This FIT KIT is testing equipment designed to facilitate the correct choice of a WalkOn AFO (ankle foot orthoses). The objective is to assess if a WalkOn Flex is appropriate for the patient. The purpose is not to evaluate the correct size.

The FIT KIT is suited for WalkOn Flex, article number 28U22. (WalkOn Flex is designed with a rubber sole attached on the footplate, which makes it easy to customize with the use of a simple pair of scissors).

The FIT KIT includes four products:

How to use the FIT KIT?

- ▶ Choose a suitable product in terms of size and side for your patient.
 - ▶ Place the AFO in the patient's shoe, preferably under any removable insole.
 - ▶ Apply the shoe and product onto the patient.
 - ▶ Let the patient walk a short distance and evaluate the effectiveness of the AFO.
 - ▶ If the result is positive, fit the patient with the appropriate size of a WalkOn Flex.
-

If the patient prefers or needs a more rigid AFO, choose a classic WalkOn 28U11 or a WalkOn Trimable 28U23 (a classic WalkOn but with a rubber sole attached on the footplate).

(WalkOn / WalkOn Trimable FIT KIT, article number 28T1N)

Note

As the size of the test model is often not entirely correct, the patient may experience some pressure at the medial malleolus. This is remedied by a correct customized product, grinding or cutting according to our fitting instructions (see the products IFU, instructions for use).

Art. no	Side	Shoe Size	Height
28U22=L/R 36-39	Left/Right	36-39	35 cm / 13,8 inch
28U22=L/R 39-42	Left/Right	39-42	36 cm / 14,2 inch
28U22=L/R 42-45	Left/Right	42-45	37,5 cm / 14,8 inch
28U22=L/R 45-48	Left/Right	45-48	39 cm / 15,4 inch

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Otto Bock HealthCare GmbH