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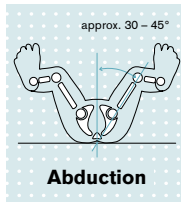
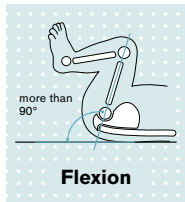
Tübingen Hip Flexion and Abduction Orthosis

Flexible and easy to handle

Dear Parents,

The Tübingen hip flexion and abduction orthosis supports the development of your child's hips. This is necessary if the child has hip dysplasia, which is the delayed development of the hip joint socket. The orthosis flexes the hips beyond 90° while slightly spreading the legs. Known as the spread-squat position, this is an entirely natural position for your baby that was already assumed in the womb.

You will find it easy to handle the orthosis. Simply read the instructions that follow.

You will be given detailed instructions on how to use the orthosis when it is first fitted by the doctor or orthotist under the doctor's responsibility.

Important

Your cooperation is crucial to the success of your child's treatment. Therefore, please read the information for parents thoroughly and follow the instructions.

A Design

The Tübingen hip flexion and abduction orthosis consists of the shoulder harness and the thigh supports, which are joined together with beaded cords.

- 1 **Shoulder harness** with hook-and-loop closure
- 2 **Thigh support** with spreader bar
- 3 **Red and white closures**
- 4 **Beaded cords, coloured** (green/white)

Red closures must be adjusted only by a doctor or the attending orthopaedic technician!

B Donning

The right way to put on the Tübingen hip flexion and abduction orthosis:

- 5 First, put on the shoulder harness from behind. Then, fasten the hook-and-loop closure so that the caterpillar shown in the illustration is visible in front. The thigh supports are lying at your child's sides.

Observe closure – The hook-and-loop closure with caterpillar must be closed at the front.

- 6 Pull the child toward your abdomen so that his or her legs are flexed at the hips. Then guide the leg supports underneath the thighs.
- 7 When the child's feet are supported on your abdomen, the beaded cords can be attached in the white closures without tension at the length prescribed by the doctor. Leave a space of about 4 cm or 2 finger widths between the upper edge of the hook-and-loop closure and the neck.
- 8 If the hook-and-loop closure is dirty, you can remove and wash it. Remove it by pressing your thumb on the protruding plastic bars at the right and left, directly inside the white plastic buckles.

Important when putting on the orthosis

The orthosis should not be in contact with bare skin. A bodysuit – preferably with a collar – is thus very practical. Garments with an opening at the crotch make it easier to change the baby's diaper.

Do not change the settings of the hip flexion and abduction orthosis prescribed by your doctor. Should any of the orthosis settings be changed unintentionally, please contact the attending doctor.

Wearing the orthosis

Naturally, your doctor will help to fit the orthosis. Then, you put the orthosis on exactly as described in steps 5 through 7 (B). Most babies get used to the new situation very quickly. If your child fights the orthosis even after familiarisation, the doctor must check the fit of the orthosis. Normally, your child should wear the hip flexion and abduction orthosis for about 23 hours a day, which means all the time except when changing and bathing. If your child occasionally lies face down, put a firm foam rubber cushion (A) under him or her.

Important in the course of treatment

If you get the impression that there is something wrong, you should contact the attending doctor immediately, regardless of when the next follow-up appointment is scheduled.

Important after the end of treatment

Regular checkups are very important even after the end of treatment.

Washing and care instructions

- The orthosis is waterproof, fully washable, corrosion-resistant and even resistant to salt water.
- The terry cloth covers and hook-and-loop closure (please remove as described in Fig. B8) are machine washable at 40 °C with standard mild detergent (no fabric softener).
- After washing, allow to air dry away from direct heat.
- An additional hook-and-loop closure and extra terry cloth covers are included in the package to use when washing or in case of wear.

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© Ottobock · 646D579=01-1403 · Technical modifications and printing errors reserved.