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## INFORMATION

Last update: 2016-03-08

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

These instructions for use provide important information on the application of the 4026 Diagonal Comfort, 4074 Diagonal Short and 4075 Diagonal Long wrist supports.

## 2 Application

### 2.1 Indications for use

The support is intended **exclusively** for treating the upper limbs and **exclusively** for contact with intact skin.

The support must be used according to the indication.

### 2.2 Indications

- Sprains, irritation and wear of the thumb saddle joint (e.g. rhizarthrosis)
- Sprains, irritation and collateral ligament injuries of the metacarpophalangeal joint (e.g. skier's thumb)

The indication must be determined by the physician.

### 2.3 Contraindications

#### 2.3.1 Absolute Contraindications

None known.

#### 2.3.2 Relative Contraindications

The following indications require consultation with a physician: skin diseases/injuries, inflammation, prominent, swollen scars, reddening and hyperthermia in the relevant area; pronounced varicose veins, especially with impaired blood return, lymph drainage disorders and unclear soft tissue swelling distal to the product; sensory and circulatory disorders in the area of the forearm and hand, e.g. associated with diabetic neuropathy.

### 2.4 Mechanism of Action

The support braces the CMC and MCP joint of the thumb.

## 3 Safety

### 3.1 Explanation of warning symbols

**⚠ CAUTION** Warning regarding possible risks of accident or injury.

**NOTICE** Warning regarding possible technical damage.

### 3.2 General safety instructions

**⚠ CAUTION**

**Contact with heat, embers or fire**

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

**⚠ CAUTION**

**Reuse on other persons and improper cleaning**

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

**NOTICE**

**Improper use and changes**

Change in or loss of functionality as well as damage to the product

- ▶ Use this product with care and only for its intended purpose.
- ▶ Do not make any improper changes to the product.

**NOTICE**

**Contact with ointments, lotions, or other products that contain oils or acids**

Insufficient stabilization due to loss of material functionality

- ▶ Do not expose the product to ointments, lotions, or other products that contain oils or acids.

## 4 Handling

### INFORMATION

- ▶ The daily duration of use and period of application are generally determined by the physician.
- ▶ The initial fitting and application of the product must be carried out by qualified personnel.
- ▶ Instruct the patient in the handling and care of the product.
- ▶ Instruct the patient to see a physician immediately if any exceptional changes are noted (e.g. worsening of the complaint).

### 4.1 Application

### ⚠ CAUTION

#### **Incorrect or excessively tight application**

Skin irritation due to overheating, local pressure points due to tight application to the body

- ▶ Ensure that the support is applied properly and fits correctly.
- ▶ Do not use the product in case of a known heat allergy.
- ▶ When in doubt, do not continue to use the product if skin irritation occurs.

### NOTICE

#### **Use of a worn or damaged product**

Limited effectiveness

- ▶ Before each use, check the product for functional reliability and for possible wear or damage.
- ▶ Do not continue using a product that is no longer functional, or that is worn or damaged.

> Open all hook-and-loop closures on the support.

1) Put the support on the thumb.

**INFORMATION: Ensure that the coil spring is parallel to the thumb.**

2) Close the hook-and-loop closure on the thumb.

3) Close the hook-and-loop closure on the wrist (see fig. 1).

4) Readjust the hook-and-loop closure on the thumb (see fig. 2).

## 4.2 Cleaning

### NOTICE

#### Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- Only clean the product with the approved cleaning agents.

Clean the support regularly:

- 1) Fasten all hook-and-loop closures.
- 2) Recommendation: Use a laundry bag or net.
- 3) Wash the support in warm water at **40 °C** (104 °F) with standard mild detergent. Do not use fabric softener. Rinse thoroughly.
- 4) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

## 5 Disposal

Dispose of the product in accordance with national regulations.

## 6 Legal Information

All legal conditions are subject to the respective national law of the country of use and may vary accordingly.

### 6.1 Liability

The manufacturer shall be liable in the event that the product is used in accordance with the descriptions and instructions in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorized modification of the product.

### 6.2 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The manufacturer therefore drew up the declaration of conformity on its own responsibility in accordance with Annex VII of the directive.

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Otto Bock HealthCare GmbH