## 4055 Manu Comfort, 4058 Manu Comfort Stable

<b>DE</b> Gebrauchsanweisung .....	4
<b>EN</b> Instructions for use .....	8
<b>FR</b> Instructions d'utilisation .....	13
<b>IT</b> Istruzioni per l'uso .....	18
<b>ES</b> Instrucciones de uso .....	23
<b>NL</b> Gebruiksaanwijzing .....	27
<b>SV</b> Bruksanvisning .....	32
<b>DA</b> Brugsanvisning .....	36
<b>NO</b> Bruksanvisning .....	41
<b>FI</b> Käyttöohje .....	45
<b>PL</b> Instrukcja użytkowania .....	50
<b>HU</b> Használati utasítás .....	54
<b>CS</b> Návod k použití .....	59
<b>HR</b> Upute za uporabu .....	63
<b>SK</b> Návod na používanie .....	67
<b>TR</b> Kullanma talimatı .....	72
<b>EL</b> Οδηγίες χρήσης .....	76
<b>RU</b> Руководство по применению .....	81
<b>JA</b> 取扱説明書 .....	86
<b>ZH</b> 使用说明书 .....	90

1

4055, 4058

2

3

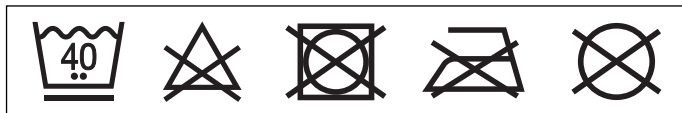
4

5

	<b>Seite / Side</b>	<b>Größe / Size</b>	<b>Handgelenkumfang / Wrist circumference</b>	
			<b>cm</b>	<b>inch</b>
	L/R	XS-S	13-16	5.1-6.3
	L/R	M-L	16-19	6.3-7.5
L/R	XL-XXL	19-23	7.5-9.0	

<b>Material</b>	PCM, Kunststoffschiene / Plastic splint, Stahlfedern / Steel springs
-----------------	---

## INFORMATION

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### INFORMATION

Date of last update: 2020-05-18

- Please read this document carefully before using the product and observe the safety notices.
- Instruct the user in the safe use of the product.
- Please contact the manufacturer if you have questions about the product or in case of problems.

- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

These instructions for use provide important information on the fitting and application of the 4055 Manu ComforT and 4058 Manu ComforT Stable wrist supports.

## 2 Intended use

### 2.1 Indications for use

The support is intended **exclusively** for treating the upper limbs and **exclusively** for contact with intact skin.

The support must be used according to the indication.

### 2.2 Indications

#### 4055 Manu ComforT

- Arthralgia
- Tendosynovitis
- Post-traumatic radius fracture
- Wrist instability
- Irritation (e.g. tendovaginitis, tendomyopathy, minor joint effusion, post-traumatic [sprains], post-operative, reactive irritation, and arthralgia)
- Sprains
- Subluxation/dislocation
- Carpal tunnel syndrome
- Tendovaginitis in the hand
- Osteoarthritis

#### 4058 Manu ComforT Stable

- Partial radial nerve paresis
- Post-traumatic condition
- Wrist drop due to paresis of the radial nerve

The indication must be determined by the physician.

### 2.3 Contraindications

#### 2.3.1 Absolute Contraindications

None known.

### 2.3.2 Relative Contraindications

The following indications require consultation with a physician: skin diseases/injuries, inflammation, prominent, swollen scars, reddening and hyperthermia in the relevant area; pronounced varicose veins, especially with impaired blood return, lymph drainage disorders and unclear soft tissue swelling distal to the device; sensory and circulatory disorders in the area of the forearm and hand, e.g. associated with diabetic neuropathy.

### 2.4 Mechanism of Action

The support stabilizes the wrist.

## 3 Safety

### 3.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

### 3.2 General safety instructions

 **CAUTION**

**Contact with heat, embers or fire**

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

 **CAUTION**

**Reuse on other persons and improper cleaning**

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

 **CAUTION**

**Use of the product on skin sensitive to heat**

Skin irritation due to over-sensitivity to heat

- ▶ Do not use the product for more than 4 hours without a break.

**NOTICE****Improper use and changes**

Change in or loss of functionality as well as damage to the product

- ▶ Use this product with care and only for its intended purpose.
- ▶ Do not make any improper changes to the product.

**NOTICE****Contact with oils, salves, lotions or other products that contain oils or acids**

Insufficient stabilization due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

## 4 Handling

**INFORMATION**

- ▶ The daily duration of use and period of application are generally determined by the physician.
- ▶ The initial fitting and application of the product must be carried out by qualified personnel.
- ▶ Instruct the patient in the handling and care of the product.
- ▶ Instruct the patient to see a physician immediately if any exceptional changes are noted (e.g. worsening of the complaint).

### 4.1 Selecting the Size

- 1) Measure the wrist circumference.
- 2) Determine the size of the support (see size chart).

### 4.2 Fitting and Application

#### 4.2.1 Adaptation

- > Open all hook-and-loop closures.
- 1) Insert the plastic splints in the support.
  - 2) Heat the plastic splints in the support.
  - 3) Apply the support to the wrist after it has cooled off slightly and close the hook-and-loop closures.
  - 4) Shape the plastic splints manually and wait until they have hardened.

## 4.2.2 Application

### CAUTION

#### **Incorrect or excessively tight application**

Skin irritation due to overheating, local pressure points due to tight application to the body

- ▶ Ensure that the product is applied properly and fits correctly.
- ▶ When in doubt, do not continue to use the product if skin irritation occurs.

- > Open all hook-and-loop closures.
- 1) Pull the support over the wrist, placing the loop over the thumb.
- 2) Fasten all hook-and-loop closures (see fig. 2, see fig. 3, see fig. 4, see fig. 5).
- The support fits snugly without pinching.

## 5 Cleaning

### NOTICE

#### **Use of improper cleaning agents**

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

- 1) Take out all removable splints.
- 2) Fasten all hook-and-loop closures.
- 3) Recommendation: Use a laundry bag or net.
- 4) Wash the support in warm water at **40 °C** with standard mild detergent. Do not use fabric softener. Rinse thoroughly.
- 5) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).
- 6) Reinsert all splints.

## 6 Disposal

Dispose of the product in accordance with national regulations.

## 7 Legal Information

All legal conditions are subject to the respective national law of the country of use and may vary accordingly.

### 7.1 Liability

The manufacturer shall be liable in the event that the product is used in accordance with the descriptions and instructions in this document. The

manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorized modification of the product.

## 7.2 CE Conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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