## SL=AL\*, AR\*, CFP\*, F\*, HAL\*, HAR\*, ME\*

<b>DE</b> Gebrauchsanweisung .....	2
<b>EN</b> Instructions for use .....	4
<b>FR</b> Instructions d'utilisation .....	7
<b>IT</b> Istruzioni per l'uso .....	10
<b>ES</b> Instrucciones de uso .....	13
<b>PT</b> Manual de utilização .....	15
<b>NL</b> Gebruiksaanwijzing .....	18
<b>SV</b> Bruksanvisning .....	21
<b>DA</b> Brugsanvisning .....	24
<b>NO</b> Bruksanvisning .....	26
<b>FI</b> Käyttöohje .....	29
<b>CS</b> Návod k použití .....	31
<b>SK</b> Návod na používanie .....	34
<b>RU</b> Руководство по применению .....	37
<b>ZH</b> 使用说明书 .....	40

## INFORMATION

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### INFORMATION

Date of last update: 2020-04-24

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

These instructions for use provide you with important information on the use of the SL=AL\*, SL=AR\*, SL=CFP\*, SL=F\*, SL=HAL\*, SL=HAR\*, SL=ME\* carbon fibre foot plate.

## 2 Indications for use

### NOTICE

#### Improper use of the foot plate

Impairment of functionality and damage due to improper use of the foot plate

- ▶ Do not use this product as a Chopart foot plate.
- ▶ Observe the usage information in the instructions for use.

The product is intended **exclusively** for orthotic fittings of the lower limbs.

## 3 Safety

### 3.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

### 3.2 General safety instructions

#### **CAUTION**

##### Reuse on another patient

Risk of injury due to loss of functionality as well as damage to the product

- ▶ Only use the product for a single patient.

#### **CAUTION**

##### Improper processing of the material

Injuries due to lack of caution when processing the material

- ▶ Observe the instructions regarding preparation for use.

## 4 Preparation for use

#### **CAUTION**

##### Sharp edges

Injuries due to cutting

- ▶ The product must be completely wrapped with covering material and protective material prior to use.
- ▶ Use caution and maintain awareness of safety when working with the product.

#### **CAUTION**

##### Hazardous dust during grinding

Injury due to defective or lack of protective equipment.

- ▶ Wear a dust mask during grinding.
- ▶ Ensure sufficient ventilation in the work area.

### NOTICE

#### Drilling or attaching fasteners on the product

Breakage or delamination of the product

- ▶ Do not drill the product.
- ▶ Do not attach fasteners on the product.

#### NOTICE

##### **Thermal overloading of the product**

Damage due to improper thermal treatment

- ▶ Thermoforming the product is prohibited.
- ▶ Do not carry out any heat treatment at a temperature above **93 °C (200 °F)**.

#### INFORMATION

##### **Use without suitable footwear**

Only use the product with closed footwear, taking the allowable effective heel height into account.

- 1) Choose the covering material according to the physician's prescription and individual patient needs.
- 2) Take an impression of the patient's foot.
- 3) Fabricate the cover using standard thermoforming techniques.
- 4) Trim the cover to fit the individual patient.
- 5) Position the cover on the carbon fibre foot plate.
- 6) Mark the outline of the plate on the cover.
- 7) Sand the cover with a belt sander or suitable device so that a projection of about **2 mm** over the edge of the plate remains.
- 8) Smooth the edge with sandpaper or steel wool.
- 9) Clean the plate using a degreasing cleaner.
- 10) Use contact adhesive to attach the plate under the cover. The edge of the cover should project **2 mm** beyond the outline of the plate.
- 11) Insert the plate into the shoe.
- 12) Perform a trial fitting with the patient. Test the fit and stiffness.
- 13) Trim the protective material for the distal side of the plate to match the dimension of the proximal cover.
- 14) Glue the protective material under the plate using contact adhesive.
- 15) Perform a trial fitting with the patient. If necessary: Adapt the product.

The "spring arch support" and "Morton's extension" designs provide medial support for the arch of the foot and a small heel depression in order to ensure correct positioning.

Carry out work steps one through 12 to cover and finish the spring arch support and Morton's extensions, but do not affix any protective material to the bottom of the foot plates.

## 5 Cleaning

Promptly clean the product after contact with water containing salt, chlorine or soap, or if it gets dirty.

- 1) Rinse the product with clear fresh water.
- 2) Dry the product with a cloth or allow it to air dry. Avoid exposure to direct heat (e. g. from an oven or radiator).

## 6 Maintenance

#### INFORMATION

##### **The product may be exposed to increased loads by the patient.**

- ▶ Shorten the maintenance intervals according to the expected loads.

## 7 Disposal

Dispose of the product in accordance with national regulations.

## 8 Warranty

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

## 9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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