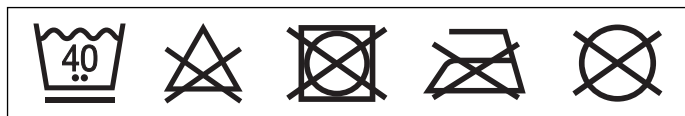
## 7126 Shoulder Support

<b>DE</b> Gebrauchsanweisung .....	3
<b>EN</b> Instructions for use .....	7
<b>FR</b> Instructions d'utilisation .....	11
<b>IT</b> Istruzioni per l'uso .....	16
<b>ES</b> Instrucciones de uso .....	20
<b>PT</b> Manual de utilização .....	24
<b>NL</b> Gebruiksaanwijzing .....	29
<b>SV</b> Bruksanvisning .....	33
<b>DA</b> Brugsanvisning .....	37
<b>NO</b> Bruksanvisning .....	41
<b>FI</b> Käyttöohje .....	46
<b>IS</b> Notkunarleiðbeiningar .....	50
<b>CS</b> Návod k použití .....	54
<b>EL</b> Οδηγίες χρήσης .....	58
<b>ET</b> Kasutusjuhhis .....	63
<b>RU</b> Руководство по применению .....	67
<b>JA</b> 取扱説明書 .....	72
<b>ZH</b> 使用说明书 .....	76

	Größe / Size	Ellenbogenumfang / Elbow circumference	
		cm	inch
	XS	24-28	9.5-11.0
	S	28-32	11.0-12.6
	M	32-35	12.6-13.8
	L	35-38	13.8-15.0
	XL	38-42	15.0-16.5

<b>Material</b>	Neopren-SBR, PA
-----------------	-----------------

## INFORMATION

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### INFORMATION

Date of last update: 2021-02-12

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

These instructions for use provide important information on the fitting and application of the 7126 shoulder support.

## 2 Intended use

### 2.1 Indications for use

The support is intended **exclusively** for orthotic fittings of the shoulder and **exclusively** for contact with intact skin.

The support must be used according to the indication.

## 2.2 Indications

- Irritation in the shoulder region
- Degeneration of the rotator cuff
- Tendomyopathy in the shoulder region
- Osteoarthritis of the shoulder

The indication must be determined by the physician.

## 2.3 Contraindications

### 2.3.1 Absolute Contraindications

None known.

### 2.3.2 Relative Contraindications

The following indications require consultation with a physician: skin diseases/injuries, inflammation, prominent scars that are swollen, reddening and hyperthermia of the fitted limb; pronounced varicose veins, especially with return flow impairment, lymphatic flow disorders, including unclear soft tissue swelling distal to the support; sensory and circulatory disorders in the area of the arm and hand.

## 2.4 Mechanism of Action

This support promotes sensorimotor function and helps activate muscular stabilization in the shoulder. The support has a warming effect and helps relieve pain.

## 3 Safety

### 3.1 Explanation of warning symbols

**CAUTION**

Warning regarding possible risks of accident or injury.

**NOTICE**

Warning regarding possible technical damage.

### 3.2 General safety instructions

**CAUTION**

#### **Contact with heat, embers or fire**

Risk of injury (such as burns) and risk of product damage

- ▶ Keep the product away from open flames, embers and other sources of heat.

**⚠ CAUTION****Reuse on other persons and improper cleaning**

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

**NOTICE****Contact with oils, salves, lotions or other products that contain oils or acids**

Insufficient stabilization due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

**NOTICE****Improper use and changes**

Change in or loss of functionality as well as damage to the product

- ▶ Use this product with care and only for its intended purpose.
- ▶ Do not make any improper changes to the product.

## 4 Handling

**INFORMATION**

- ▶ The daily duration of use and period of application are generally determined by the physician.
- ▶ The initial fitting and application of the product must be carried out by qualified personnel.
- ▶ Instruct the patient in the handling and care of the product.
- ▶ Instruct the patient to see a physician immediately if any exceptional changes are noted (e.g. worsening of the complaint).

### 4.1 Selecting the Size

- 1) Measure the circumference of the upper arm.
- 2) Determine the size of the support (see size chart).

## 4.2 Application

### CAUTION

#### **Use of a product with neoprene**

Allergic reactions due to neoprene or heat allergies

- ▶ Do not wear products with neoprene for more than 3 – 4 hours without interruption (unless explicitly instructed otherwise by the doctor).
- ▶ Do **not** use the product in case of a known neoprene or heat allergy.

### CAUTION

#### **Incorrect or excessively tight application**

Skin irritation due to overheating, local pressure points due to tight application to the body

- ▶ Ensure that the product is applied properly and fits correctly.
- ▶ When in doubt, do not continue to use the product if skin irritation occurs.

### NOTICE

#### **Use of a worn or damaged product**

Limited effectiveness

- ▶ Before each use, check the product for functional reliability and for possible wear or damage.
- ▶ Do not continue using a product that is no longer functional, or that is worn or damaged.

- > Open all hook-and-loop closures on the support.
- 1) Pull the support over the arm (see fig. 1).
- 2) Feed the long strap under the opposite arm and fasten it in front of the chest (see fig. 2).
- 3) Close the hook-and-loop closure on the upper arm.

## 4.3 Cleaning

### NOTICE

#### **Use of improper cleaning agents**

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

Clean the support regularly:

- 1) Fasten all hook-and-loop closures.
- 2) Recommendation: Use a laundry bag or net.

- 3) Wash the support in warm water at **40 °C** (104 °F) with standard mild detergent. Do not use fabric softener. Rinse thoroughly.
- 4) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

## 5 Disposal

Dispose of the product in accordance with national regulations.

## 6 Legal Information

All legal conditions are subject to the respective national law of the country of use and may vary accordingly.

### 6.1 Liability

The manufacturer shall be liable in the event that the product is used in accordance with the descriptions and instructions in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorized modification of the product.

### 6.2 CE Conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

### 6.3 Local Legal Information

Legal information that applies **exclusively** to specific countries is written in the official language of the respective country of use after this chapter.

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