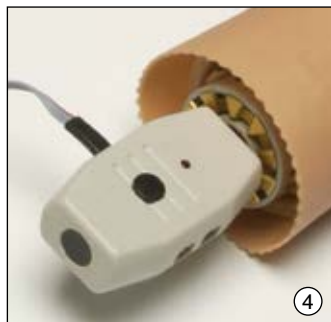
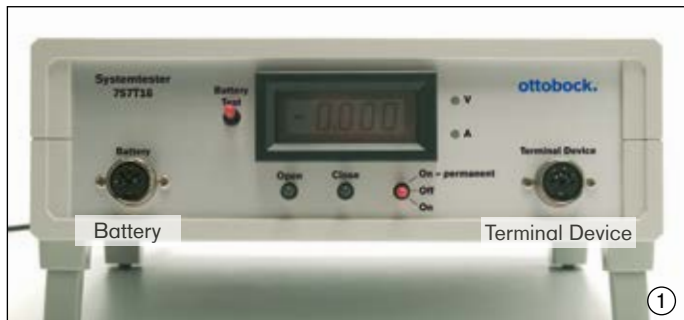
## 757P28

<b>DE</b>	Gebrauchsanweisung .....	3
<b>EN</b>	Instructions for use .....	6
<b>FR</b>	Instructions d'utilisation .....	9
<b>IT</b>	Istruzioni per l'uso .....	12
<b>ES</b>	Instrucciones de uso .....	15
<b>NL</b>	Gebruiksaanwijzing .....	18
<b>SV</b>	Bruksanvisning .....	21
<b>NO</b>	Bruksanvisning .....	24

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**Last update: 2020-11-02**

**If assistance is required during the start-up, use or maintenance of the product, as well as in the event of unexpected operating behaviour or circumstances, contact the manufacturer (see manufacturer's address on the back of this accompanying document).**

## 1 Intended use

The Ottobock 757P28 Test Plug is **solely** to be used for functional testing of System Electric Hands in combination with the System Tester 757T16/757T1 and System Electric Greifers of the MYOBOCK system prostheses with Dynamic Mode Control (DMC/DMC plus).

The following Ottobock components can be tested:

- 8E38=6\*, 8E39=6\*, 8E41=6\*, 8E44=6\* System Electric Hands DMC
- 8E33=9, 8E34=9 System Electric Greifers DMC VariPlus
- 8E38=8\*, 8E39=8\*, 8E41=8\* SensorHand Speed
- 8E33=7, 8E34=7 System Electric Greifers Digital Twin

## 2 Safety instructions

- The test plug can only be put into operation in conjunction with 757T16/757T1 System Tester.
- The test plug may only be used in dry rooms.
- Do not allow debris or liquids to get into the test plug.
- The test plug should not be subjected to intense smoke, dust, mechanical vibrations, shocks or high temperatures.
- Check the test plug for any visible damage before every use. In case of damage the apparatus should immediately be sent to an Ottobock service facility.
- Tests with 757P28 Test Plug may only be performed by technician that has attended to the respective seminar.
- Avoid using the device near strong magnetic or electric interference sources such as transformer stations or transmitters.

- The test plug and test plug components may be opened or repaired only by authorized Ottobock Service technicians.
- No liability will be assumed if the test plug is operated incorrectly or used for purposes other than as originally intended.

### 3 Operation

For testing of a component proceed as follows::

1. Put all switches of the 757T16 System Tester in mid-position.
2. Connect the accumulator via appropriate cable to the 'Battery' receptacle (Fig. 1). Important for the test: Use fully charged batteries!
3. Connect 757P28 Test Plug to the receptacle marked with 'Terminal Device' (Fig. 1) and to the component to be tested.
4. Keep the enabling button pressed while pressing one of both function keys simultaneously: Hand/Greifer should open or close (Fig. 2).

#### 3.1 LED is lit (Fig. 3)

- The switch of the test plug is on.
- The maximum gripping speed and grip force are generated during the functional test run.

#### 3.2 LED is off (Fig. 4)

- Test plug switch is off.
- System Electric Hand/Electric Greifer close with minimum gripping speed and grip force.

**To ensure that the tests of the MYOBOCK components are performed correctly and above all to be able to apply the results, it is necessary to attend to a MYOBOCK Service Seminar at Ottobock that is offered upon request.**

### 4 Disposal

These products may not be disposed of with household waste in some jurisdictions. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in your country regarding return and collection processes.

## **5 Trademarks**

All denotations within this accompanying document are subject to the provisions of the respective applicable trademark laws and the rights of the respective owners, with no restrictions.

All brands, trade names or company names may be registered trademarks and are subject to the rights of the respective owners.

Should trademarks in this accompanying document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

## **6 Liability**

Otto Bock Healthcare Products GmbH, hereafter referred to as manufacturer, assumes liability only if the user complies with the processing, operating and maintenance instructions as well as the service intervals. The manufacturer explicitly states that this product may only be used in combination with components that were authorized by the manufacturer (see instructions for use and catalogs). The manufacturer does not assume liability for damage caused by component combinations which it did not authorize.

The product may only be opened and repaired by authorized Ottobock technicians.

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