## Socket adapters

<b>DE</b> Gebrauchsanweisung .....	4	<b>CS</b> Návod k použití.....	66
<b>EN</b> Instructions for use .....	8	<b>RO</b> Instrucțiuni de utilizare.....	71
<b>FR</b> Instructions d'utilisation.....	13	<b>HR</b> Upute za uporabu .....	76
<b>IT</b> Istruzioni per l'uso .....	18	<b>SL</b> Navodila za uporabo .....	81
<b>ES</b> Instrucciones de uso .....	23	<b>SK</b> Návod na používanie .....	86
<b>PT</b> Manual de utilização.....	28	<b>BG</b> Инструкция за употреба .....	90
<b>NL</b> Gebruiksaanwijzing.....	33	<b>TR</b> Kullanma talimatı .....	96
<b>SV</b> Bruksanvisning .....	38	<b>EL</b> Οδηγίες χρήσης .....	100
<b>DA</b> Brugsanvisning .....	43	<b>RU</b> Руководство по применению .....	105
<b>NO</b> Bruksanvisning .....	47	<b>JA</b> 取扱説明書.....	111
<b>FI</b> Käyttöohje.....	52	<b>ZH</b> 使用说明书.....	115
<b>PL</b> Instrukcja użytkownika.....	57	<b>KO</b> 사용 설명서.....	119
<b>HU</b> Használati utasítás .....	62		

1

2

3

4

5

6

7

# 1 Produktbeschreibung

Deutsch

## INFORMATION

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### INFORMATION

Date of last update: 2020-05-04

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.

► Please keep this document for your records.

### 1.1 Construction and Function

Socket adapters serve as the connection between prosthetic sockets and other compatible components of modular prostheses. They have a four-hole connection and various distal connectors. These instructions for use apply to the following socket adapters:

Reference number	Connection
4R23	Pyramid adapter (see fig. 1)
4R54, 4R74, 4R74=AL	Pyramid adapter with bore (see fig. 2)
4R73=A	Pyramid adapter, 7 mm axial offset (see fig. 3)
4R73=D	Pyramid adapter, 5 mm diagonal offset (see fig. 4)
4R77	Pyramid adapter, rotating, with bore (see fig. 5)
4R22, 4R55, 4R95	Pyramid receiver (see fig. 6)
4R37, 4R51	Pyramid receiver, rotating (see fig. 7)

### 1.2 Combination possibilities

This prosthetic component is compatible with Ottobock's system of modular connectors. Functionality with components of other manufacturers that have compatible modular connectors has not been tested.

## 2 Intended use

### 2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

### 2.2 Area of application

- The maximum approved body weight is specified in the technical data (see Page 13).

## 2.3 Environmental conditions

### Storage and transport

Temperature range  $-20\text{ }^{\circ}\text{C}$  to  $+60\text{ }^{\circ}\text{C}$  ( $-4\text{ }^{\circ}\text{F}$  to  $+140\text{ }^{\circ}\text{F}$ ), relative humidity 20 % to 90 %, no mechanical vibrations or impacts

### 4R73=\*

#### Allowable environmental conditions

**Temperature range:**  $-10\text{ }^{\circ}\text{C}$  to  $+45\text{ }^{\circ}\text{C}$

**Moisture:** relative humidity: 20% to 90%, non-condensing

#### Unacceptable environmental conditions

**Chemicals/liquids:** fresh water, salt water, perspiration, urine, acids, soapsuds, chlorine water

**Solids:** dust, sand, highly hygroscopic particles (e. g. talcum)

### 4R22, 4R23, 4R37, 4R51, 4R55 4R74=\*, 4R95

#### Allowable environmental conditions

**Temperature range:**  $-10\text{ }^{\circ}\text{C}$  to  $+45\text{ }^{\circ}\text{C}$

**Moisture:** relative humidity: 20% to 90%, non-condensing

**Chemicals/liquids:** fresh water as dripping water, occasional contact with salty air (e.g. near the ocean)

**Solids:** dust

#### Prohibited environmental conditions

**Chemicals/moisture:** salt water, perspiration, urine, acids, soapsuds, chlorine water

**Solids:** dust in high concentrations (e.g. construction site), sand, highly hygroscopic particles (e.g. talcum)

### 4R77

#### Allowable environmental conditions

**Temperature range:**  $-10\text{ }^{\circ}\text{C}$  to  $+45\text{ }^{\circ}\text{C}$

**Chemicals/liquids:** fresh water, soapsuds, chlorine water

#### Allowable environmental conditions

**Moisture:** Submersion: max. 1 h in 2 m depth, relative humidity: no restrictions

**Solids:** Dust, occasional contact with sand

**Clean the product after contact with humidity/chemicals/solids, in order to avoid increased wear and damage** (see Page 12).

#### Prohibited environmental conditions

**Solids:** highly hygroscopic particles (e.g. talcum), dust in high concentrations (e.g. construction site), intensive contact with sand

**Chemicals/liquids:** salt water, perspiration, urine, acids, continuous use in liquid media

#### 4R54

#### Allowable environmental conditions

**Temperature range:** -10 °C to +45 °C (14 °F to 113 °F)

**Chemicals/liquids:** fresh water, salt water, perspiration, urine, soapsuds, chlorine water

**Moisture:** Submersion: max. 1 h in 3 m depth, relative humidity: no restrictions

**Solids:** Dust, occasional contact with sand

**Clean the product after contact with humidity/chemicals/solids, in order to avoid increased wear and damage** (see Page 12).

#### Unallowable environmental conditions

**Solids:** Highly hygroscopic particles (e.g. talcum), continuous contact with sand

**Chemicals/liquids:** Acids, continuous use in liquid media

#### 2.4 Service life

In general, all modular adapters are tested by the manufacturer for 3 million load cycles. Depending on the user's activity level, this corresponds to a maximum service life of 5 years.

### 3 Safety

#### 3.1 Explanation of warning symbols

Warning regarding possible risks of accident or injury.

Warning regarding possible technical damage.

#### 3.2 General safety instructions

##### Excessive strain on the product

Risk of injury due to breakage of load-bearing components

- ▶ Use the prosthetic components according to the MOBIS classification.

##### Unallowable combination of prosthetic components

Risk of injury due to breakage or deformation of the product

- ▶ Only combine the product with prosthetic components that are approved for that purpose.
- ▶ Based on the instructions for use of the prosthetic components, verify that they may be combined with each other.

##### Use under unallowable environmental conditions

Risk of injury due to damage to the product

- ▶ Do not expose the product to unallowable environmental conditions.
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.

- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop etc.).

**⚠ CAUTION**

**Exceeding the service life**

Risk of injury due to change in or loss of functionality and damage to the product

- ▶ Ensure that the approved service life is not exceeded.

**⚠ CAUTION**

**Mechanical damage to the product**

Risk of injury due to change in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, do not continue using the product (see "Signs of changes in or loss of functionality during use" in this section).
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

**Signs of changes in or loss of functionality during use**

Among other factors, changes in functionality can be indicated by an altered gait pattern, a change in the positioning of the prosthetic components relative to each other and by the development of noises.

**4 Scope of delivery**

Fig.	Quantity	Designation	Reference number
–	1	Instructions for use	–
–	1	Socket adapter	–
<b>For 4R22, 4R37, 4R51, 4R55</b>			

Fig.	Quantity	Designation	Reference number
–	4	Set screw	506G3=M8x12-V
<b>For 4R37, 4R51, 4R77</b>			
–	1	Pressure plate	4Y19
<b>For 4R73=A, 4R73=D</b>			
–	4	Countersunk head screw	501S41=M6x12x11.4
–	4	Countersunk head screw	501S41=M6x25x11.4
–	4	Countersunk head screw	501S41=M6x30x11.4
<b>For 4R95</b>			
–	4	Set screw	506G3=M8x14

**5 Preparing the product for use**

**⚠ CAUTION**

**Incorrect alignment or assembly**

- Risk of injury due to damaged prosthetic components
- ▶ Observe the alignment and assembly instructions.

**⚠ CAUTION**

**Improper assembly of the screw connections**

Risk of injury due to breakage or loosening of the screw connections

- ▶ Clean the threads before every installation.
- ▶ Apply the specified torque values.
- ▶ Follow the instructions regarding the length of the screws and about how to secure the screws.

## 5.1 Mounting the adapter

- Install the 4R73=A\* and 4R73=D\* socket adapters with the supplied countersunk head screws. Use the following selection table for this purpose.

Connecting component	Compatible countersunk head screw
5R2 Lamination disc	501S41=M6x12x11.4
4R101 Sliding adapter	
5R1* Socket attachment block	501S41=M6x25x11.4
5R6=* Modular adapter	501S41=M6x30x11.4

> **Required materials:** 710D20 torque wrench, 636K13 Loctite 241

- 1) **If the pyramid receiver or the pyramid adapter is rotatable:** Place the pressure plate on the adapter.
- 2) Place the adapter on the connecting component.
- 3) **Optional:** Align the pyramid receiver or the pyramid adapter.
- 4) Secure the screws with Loctite.
- 5) Screw in the 2 posterior countersunk head screws and tighten them (tightening torque: **12 Nm**).
- 6) Screw in the 2 anterior countersunk head screws and tighten them (tightening torque: **12 Nm**).

## 5.2 Connecting the pyramid adapter and pyramid receiver

> **Required materials:** torque wrench (e.g. 710D20), 636K13 Loctite 241

- 1) **Fitting:**  
Screw in the set screws.  
Use the torque wrench to tighten the set screws (**10 Nm**).
- 2) **Definitive assembly:**  
Secure the set screws with Loctite.  
Screw in the set screws.  
Pre-tighten the set screws with the torque wrench (**10 Nm**) and then tighten them (**15 Nm**).
- 3) Replace any set screws that are protruding or are recessed too much with appropriate set screws (see selection chart).

Selection table for set screws

Reference number	Length (mm)
506G3=M8X12-V	12
506G3=M8X14	14
506G3=M8X16	16

## 6 Cleaning

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

## 7 Maintenance

- A visual inspection and functional test of the prosthetic components should be performed after the first 30 days of use.
- Inspect the entire prosthesis for wear during normal consultations.
- Conduct annual safety inspections.

## 8 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

## 9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

## 9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

## 9.3 Warranty

The manufacturer warrants this device from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

## 10 Technical data

Reference number	4R22	4R23	4R37	4R51
Weight [g]	85	85	140	80
System height [mm]	33	-11	36	36
Material	Stainless steel			Titanium
Max. body weight [kg]	125			150

Reference number	4R54	4R55	4R73=A	4R73=D
Weight [g]	50	50	115	115
System height [mm]	-11	33	-11	-11
Material	Titanium			
Max. body weight [kg]	150			

Reference number	4R74	4R74=AL	4R77	4R95
Weight [g]	55	70	70	50
System height [mm]	-7	-7	-9	33
Material	Aluminium		Titanium	Aluminium
Max. body weight [kg]	100	136	150	100

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