
7G3, 7U2*, 7U10, 7U15*, 7U25, 7U26, 7U27,
7U30*, 7U32*, 7U33, 7U42, 7U43, 7U46

| | | |
|-----------|----------------------------------|----|
| DE | Gebrauchsanweisung | 3 |
| EN | Instructions for use | 9 |
| FR | Instructions d'utilisation | 15 |
| IT | Istruzioni per l'uso | 22 |
| ES | Instrucciones de uso | 29 |
| PT | Manual de utilização | 35 |
| NL | Gebruiksaanwijzing | 42 |
| SV | Bruksanvisning | 48 |
| DA | Brugsanvisning | 54 |
| NO | Bruksanvisning | 60 |
| PL | Instrukcja użytkowania | 66 |
| HU | Használati utasítás | 73 |
| CS | Návod k použití | 79 |
| TR | Kullanma talimatı | 85 |
| EL | Οδηγίες χρήσης | 91 |
| RU | Руководство по применению | 98 |

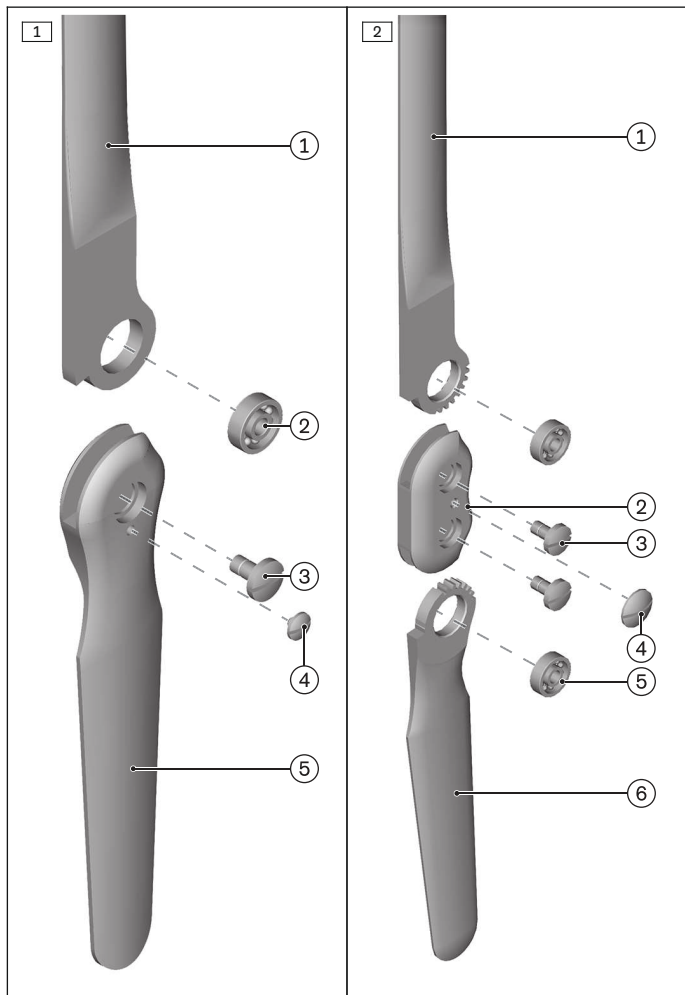INFORMATION

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INFORMATION

Date of last update: 2021-09-23

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

These instructions for use provide you with important information on the processing of the 7G3, 7U2*, 7U10, 7U15*, 7U25, 7U26, 7U27, 7U30*, 7U32*, 7U33, 7U42, 7U43 and 7U46 lower leg joint bars.

2 Product description

The lower leg knee joint bars are suitable for the fabrication of a transtibial prosthesis with a thigh sleeve.

2.1 Available sizes

| Lower leg joint bars | | | |
|----------------------|----------------------------------|-------------|-----------------|
| Reference number | Material | Width in mm | Thickness in mm |
| 7G3 | Orthopaedic steel, nickel-plated | 23 | 5 |
| 7U2* | Orthopaedic steel | 24 | 3 |
| 7U10 | Orthopaedic steel | 24 | 3 |
| 7U15* | Orthopaedic steel, nickel-plated | 24 | 3 |
| 7U25 | Orthopaedic steel, nickel-plated | 24 | 3 |
| 7U26 | Orthopaedic steel, nickel-plated | 22 | 2.5 |
| 7U27 | Stainless steel | 20 | 3 |
| 7U30* | Stainless steel | 20 | 2.3 |
| 7U32* | Stainless steel | 20 | 2.3 |
| 7U33 | Stainless steel | 20 | 2.3 |
| 7U42 | Stainless steel | 20 | 2.3 |
| 7U43 | Orthopaedic steel | 22 | 2.5 |
| 7U46 | Stainless steel | 20 | 1.75 |

2.2 Components

| (see fig. 1) | | | | | |
|------------------|----------------------------------|-----------------------|----------------------|---------------------|----------------------------------|
| Reference number | Upper joint bar section (item 1) | Ball bearing (item 2) | Joint screw (item 3) | Lock screw (item 4) | Lower joint bar section (item 5) |
| 7G3 | | 509K11* | 501A22 | 501S6* | |
| 7U2* | 7A1* | 509K11* | 501A25 | 501S22* | 7B3* |
| 7U15* | 7A11* | 509K11* | 501A22 | 501S6* | 7B13* |
| 7U25 | 7A11* | 509K11* | 501A22 | 501S6* | 7B13* |

| (see fig. 1) | | | | | |
|------------------|----------------------------------|-----------------------|----------------------|---------------------|----------------------------------|
| Reference number | Upper joint bar section (item 1) | Ball bearing (item 2) | Joint screw (item 3) | Lock screw (item 4) | Lower joint bar section (item 5) |
| 7U26 | 7A12* | 509K11* | 501A22 | 501S6* | 7B12* |
| 7U30* | 7A5* | 509K11* | 501A6 | 501S22* | 7B5* |
| 7U32* | 7A6* | 509K11* | 501A6 | 501S22* | 7B5* |
| 7U33 | 7A5* | 509K11* | 501A6 | 501S22* | 7B5* |
| 7U42 | 7A6* | 509K11* | 501A6 | 501S22* | 7B5* |
| 7U43 | 7A3* | 509K11* | 501A25 | 501S22* | 7B4* |
| 7U46 | 7A14* | 509K15 | 501A12 | 501S22* | 7B7* |

| (see fig. 2) | | | | | | |
|------------------|----------------------------------|----------------|----------------------|---------------------|-----------------------|----------------------------------|
| Reference number | Upper joint bar section (item 1) | Joint (item 2) | Joint screw (item 3) | Lock screw (item 4) | Ball bearing (item 5) | Lower joint bar section (item 6) |
| 7U10 | 7A9* | 7Y12 | 501A24* | 501S10 | 509K11* | 7A8* |

| (see fig. 2) | | | | | | |
|------------------|----------------------------------|----------------|----------------------|---------------------|---------------------------|----------------------------------|
| Reference number | Upper joint bar section (item 1) | Joint (item 2) | Joint screw (item 3) | Lock screw (item 4) | Bushing (not illustrated) | Lower joint bar section (item 6) |
| 7U27 | 7A10* | 7Y13 | 501A32 | 501S22* | 7Y14** | 7B10* |

3 Intended use

3.1 Indications for use

The product is intended **exclusively** for use in pairs for prosthetic fittings of the lower limbs on one patient.

3.2 Indications

- For unilateral or bilateral amputation
 - Orthopaedic diseases of the lower limbs
- Indications must be determined by the physician.

3.3 Lifetime

The product is designed for a lifetime of **3 years** when used as intended and assembled professionally.

3.4 Qualification

Patients may be fitted with the product only by trained qualified personnel. The qualified personnel must be familiar with the handling of the various techniques, materials, machines and tools.

4 Safety

4.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

4.2 Safety instructions

 **CAUTION**

Excessive strain on load-bearing components

Injuries due to changes in or loss of functionality

- ▶ Only use the product for the defined area of application.
- ▶ If the product has been exposed to extreme strain (e.g. due to falling), take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

 **CAUTION**

Incorrect alignment, assembly or adjustment

Risk of injury due to change in or loss of functionality

- ▶ Assembly, adjustment and maintenance operations may only be completed by qualified personnel.
- ▶ Observe the alignment, assembly and adjustment instructions.

 **CAUTION**

Excessive strain due to use on more than one patient

Risk of injury and loss of functionality as well as damage to the product

- ▶ Use the product on only one patient.
- ▶ Observe the maintenance recommendations.

 **CAUTION**

Mechanical damage to the product

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.

- ▶ Check the product for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, discontinue use of the product and have it checked by authorised, qualified personnel.

NOTICE

Exposure of the product to unsuitable environmental conditions

Damage, brittleness or destruction due to improper handling

- ▶ Avoid storage in condensing ambient humidity.
- ▶ Avoid contact with abrasive substances (e.g. sand, dust).
- ▶ Do not expose the product to temperatures below -10 °C (14 °F) or above +60 °C (140 °F) (e.g. sauna, excessive sunlight, drying on a radiator).

5 Preparing the product for use

⚠ CAUTION

Repeated bending at the same position

Risk of injury due to breakage of the bar, change in or loss of functionality

- ▶ Avoid repeated bending at the same position.

NOTICE

Lack of parallel alignment of the joints

Mechanical wear and tear

- ▶ Use the 743R3 or 743R5 parallel alignment tool.

INFORMATION

Corrosion protection of the bars

The bars are made of corrosion and acid-resistant steel. Fine grinding and polishing improve corrosion protection of the bars.

Ottobock recommends the 618T40 sintering powder for coating.

Shaping the bars

- > **Required tools:** 743R3 or 743R5 parallel alignment tool, 711S5 bending irons.
- 1) Check the alignment of the bars on the model.
- 2) Disassemble the joint.
- 3) Adapt the bars with the bending irons.

NOTICE: Do not damage the surface hardened toothed segments of the 7U10* and 7U27* joints.

- 4) Check the adaptation on the model.
- 5) Repeat the steps until the desired result is obtained.
- 6) Remove grooves and burrs by grinding.

Finishing the surfaces of the 7U46 bars

- > The 7U46 knee joint bars can be used in a waterproof walking aid.
- ▶ Polish the surfaces of the bars on all sides.

Improving corrosion protection

Polish or sinter the product for improved corrosion protection. Ottobock recommends the 618T40* sintering powder for surface coating.

- ▶ Do not heat the material longer than **5 minutes** at max. **150 °C** for surface coating.

Using protective devices

- ▶ Use joint protectors to protect the product against soiling.

6 Cleaning

Promptly clean the product after contact with water containing salt, chlorine or soap, or if it gets dirty.

- 1) Rinse the product with clear fresh water.
- 2) Dry the product with a cloth or allow it to air dry. Avoid exposure to direct heat (e. g. from an oven or radiator).

7 Maintenance

INFORMATION

The product may be exposed to increased loads by the patient.

- ▶ Shorten the maintenance intervals according to the expected loads.

The manufacturer requires at least a semi-annual inspection of the product to verify functionality and check for wear.

Only use 633F7 special lubricant.

Replacing the ball bearing

NOTICE

Canting the ball bearing

Mechanical damage

- ▶ Do not cant or damage the ball bearing while inserting it into the orthotic joint.

- 1) Loosen the screws and disassemble the orthotic joint.

- 2) Remove the ball bearing from the orthotic joint and replace it with a new one.
- 3) Assemble the orthotic joint and tighten the screws.
- 4) **Lock screw:** Clean the threads of the screw connections using a degreasing cleaner and secure them with Loctite 241.

8 Disposal

Dispose of the product in accordance with national regulations.

9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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