DE 617H46 Haftvermittler

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EN 617H46 Bonding agent

For bonding and repairing vulcanised silicone rubber and for sealing valves

Preparing the surface:

The surface should be thoroughly cleaned, free of grease and dry.

Applying the bonding agent:

Apply a thin layer to both surfaces to be bonded, working quickly and avoiding bubbles.

Processing:

Press together the surfaces to be bonded to create good contact between them.

Shaping:

A skin will form after one minute. A wetted modelling instrument or an ice cube can be used for shaping before the skin forms.

Setting time:

The setting time depends on the wall thickness, the thickness of the adhesive layer and the relative humidity. Setting is more rapid at higher humidity. The setting time is approx. 12 h to 24 h at 21 °C and 30 per cent to 60 per cent humidity.

Storage:

Close the tube tightly and store in a cool, dry place.

⚠ CAUTION

Avoid contact of the unset adhesive with the skin and eyes. Acetic acid is released during setting and can cause eye irritation. Wear protective goggles. Keep out of the reach of children. In case of eye contact, rinse the eye thoroughly under running water and call a doctor. In case of skin contact, remove the adhesive with a dry piece of cloth or tissue.

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