
# ottobock.

## 10V18,10V36

<b>DE</b>	Gebrauchsanweisung .....	3
<b>EN</b>	Instructions for use .....	8
<b>FR</b>	Instructions d'utilisation .....	13
<b>IT</b>	Istruzioni per l'uso .....	18
<b>ES</b>	Instrucciones de uso .....	23
<b>PT</b>	Manual de utilização.....	28
<b>NL</b>	Gebruiksaanwijzing.....	33
<b>SV</b>	Bruksanvisning .....	38
<b>DA</b>	Brugsanvisning.....	42
<b>NO</b>	Bruksanvisning .....	47
<b>PL</b>	Instrukcja użytkowania .....	52
<b>CS</b>	Návod k použití.....	57
<b>RO</b>	Instrucțiuni de utilizare.....	61
<b>HR</b>	Upute za uporabu .....	67
<b>TR</b>	Kullanma talimatı.....	72
<b>EL</b>	Οδηγίες χρήσης .....	76
<b>RU</b>	Руководство по применению .....	82
<b>JA</b>	取扱説明書.....	87
<b>ZH</b>	使用说明书.....	92

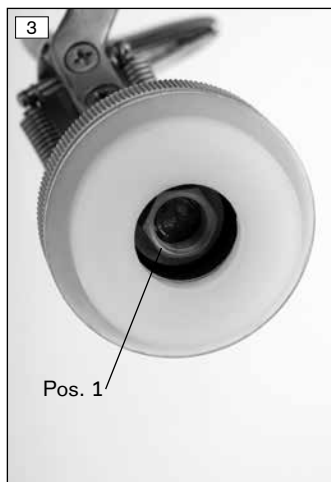## INFORMATION

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## INFORMATION

Last update: 2021-02-16

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

## Explanation of symbols

**⚠ WARNING** Warnings regarding possible risks of severe accident or injury.

**⚠ CAUTION** Warnings regarding possible risks of accident or injury.

**NOTICE** Warnings regarding possible technical damage.

**INFORMATION** Additional information on the fitting/use.

The scope of delivery is shown on the front page.

## 1 Single components

### 1.1 Single components

For available single components refer catalogue.

## 2 Description

### 2.1 Indications for use

The Ottobock 10V18=\*/10V36=\* wrist joints are intended **exclusively** for the exoprosthetic fitting of the upper limbs in combination with Ottobock arm components.

## 2.2 Field of application

The Ottobock 10V18=\* / 10V36=\* wrist joints can be used with an Ottobock cable-activated hook or an Ottobock system hand.

## 2.3 Lifetime

Expected lifetime: 5 years

The design, manufacturing and requirements for the intended use of the product are based on the expected lifetime.

## 2.4 Safety instructions

Please forward the following safety instructions to your patients:

### WARNING

**Risk of accident when driving a motor vehicle.** An upper extremity amputee's ability to drive a vehicle is determined on a case-by-case basis. Factors include the type of fitting (amputation level, unilateral or bilateral, residual limb conditions, design of the prosthesis) and the amputee's abilities.

All persons are required to observe their country's national and state driving laws when operating motor vehicles. For insurance purposes, drivers should have their driving ability examined and approved by an authorized test center.

For maximum safety and convenience, Ottobock recommends that, at the very least, a specialist evaluate the need for any adaptations to the car (such as by installing a steering fork, automatic shift). Risk-free driving should be ensured even when the prosthesis is not functioning.

### CAUTION

**Risk of injuries caused by overuse.** The Ottobock prosthetic components described in this manual have been developed for everyday life activities and must not be used for unusual activities such as extreme sports (free climbing, paragliding, etc.).

Careful handling of the prosthesis and its components not only increases their service life but, above all, ensures the patient's safety!

Should the prosthesis be subjected to unusual stresses (such as a fall), immediately contact your prosthetist and have the prosthesis inspected for any damage. If necessary, the responsible prosthetist will pass the prosthesis on to the Ottobock Service.

#### NOTICE

**Risk of corrosion.** Please avoid exposing prosthetic components to surroundings that corrode metal parts, for example, freshwater, saltwater, acids and other liquids.

Using this medical product in such environmental conditions will render all claims against Otto Bock HealthCare null and void.

#### NOTICE

**Risk of damage due to improper environmental conditions.** The prosthesis and its components should not be subjected to intense smoke, dust, vibrations, shocks or high temperatures. Do not allow debris or liquids to get into the prosthesis and its components.

Failure to follow these instructions can lead to malfunctioning and damage of the prosthesis.

#### NOTICE

**Risk of damage due to use of improper cleansing agents.** Do not use any aggressive cleaning agents since they could cause damage to bearings, seals and plastic parts.

## 2.5 Function

### 10V18=\* / 10V36=\*

The Ottobock Friction Wrist Units are available in the following designs:

- 10V18=\* with metric thread M12×1.5
- 10V36=\* with thread measured in inches ½"-20

For available sizes see table in section 3.

The Ottobock Friction Wrist Units provide connection between prosthetic hand and forearm socket.

The terminal device can be stopped in any rotational position through the deformation of the Rubber Friction Ring 11D27=\*.

### 3 Technical data

Article number	10V18=	10V18= 10V36=		
	40	34	45	50
Weight (g)	50	35	55	65
Total height (mm)	34	30	36	

### 4 Handling

#### 4.1 Assembly

##### NOTICE

**Damage due to improper use of adhesive.** To guarantee a solid bond, clean the respective areas with acetone before applying the adhesive.

Do not touch the areas after you have degreased them and when the acetone has dried up, apply the adhesive immediately.

##### INFORMATION

Follow all processing and safety instructions by the adhesive manufacturer.

#### 10V18=\*/10V36=\*

The laminated forearm socket must be fabricated with the appropriate lamination form.

The four bores in the lamination ring must be transferred into forearm component.

Bond the lamination ring in the forearm socket with Sealing Resin Compact Glue 636K18 and let the bonding dry. Subsequently assemble the wrist unit and attach the unit with the 4 oval countersunk self-tapping screws.

The system hands or cable activated hooks are screwed into the wrist joint directly.

If you use the wrist unit in conjunction with an Ottobock cable activated

hook, it will be installed with the Friction Plate 10A46=\*/10A53=33 (fig. 1, item 1).

If you use the wrist unit in conjunction with an Ottobock system hand, abstain from using the Friction Plate (fig. 2).

## **5 Disposal**

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

## **6 Legal information**

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### **6.1 Liability**

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

### **6.2 CE conformity**

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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