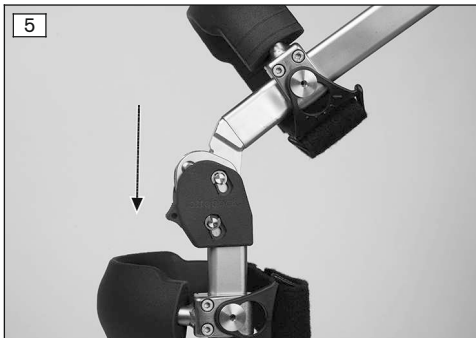
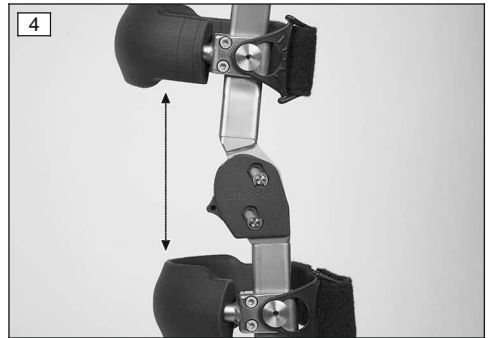
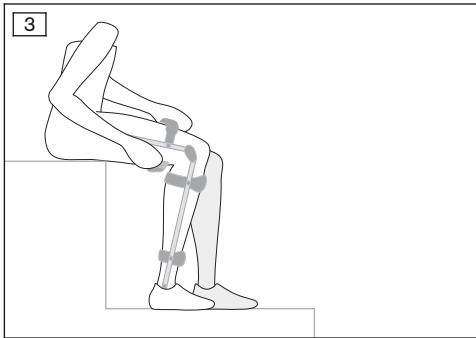
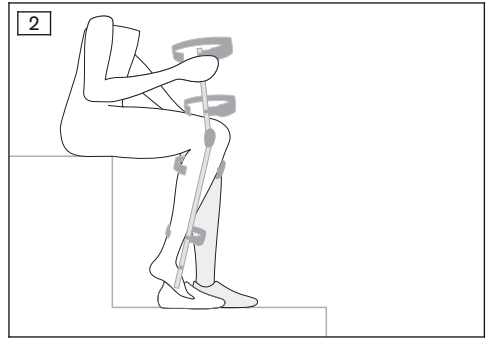
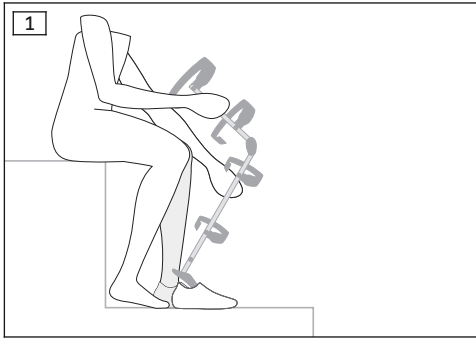
## 170K1=80 Free Walk, 170K1=120 Free Walk

<b>DE</b> Gebrauchsanweisung (Benutzer) .....	3
<b>EN</b> Instructions for use (user) .....	7
<b>FR</b> Instructions d'utilisation (Utilisateur) .....	10
<b>IT</b> Istruzioni per l'uso (Utilizzatore) .....	15
<b>ES</b> Instrucciones de uso (Usuario) .....	19
<b>PT</b> Manual de utilização (Usuário) .....	23
<b>NL</b> Gebruiksaanwijzing (Gebruiker) .....	26
<b>SV</b> Bruksanvisning (Användare) .....	31
<b>DA</b> Brugsanvisning (Bruger) .....	34
<b>NO</b> Bruksanvisning (Bruker) .....	38
<b>FI</b> Käyttöohje (Käyttäjä) .....	42
<b>TR</b> Kullanma talimatı (Kullanıcı) .....	46

## INFORMATION

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## INFORMATION

Date of last update: 2020-04-14

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

These instructions for use provide you with important information on the use of the 170K1=80 and 170K1=120 Free Walk orthosis system.

## 2 Product description

### 2.1 Design

The Ottobock Free Walk orthoses include an orthosis knee joint with lock. 3 modes can be selected on the orthosis knee joint:

- **Free Walk mode:** The knee joint locks in the stance phase and unlocks in the swing phase.
- **Permanent lock:** The knee joint is permanently locked.
- **Free motion mode:** The knee joint is permanently unlocked.

## 3 Intended use

### 3.1 Indications for use

The orthosis is intended **exclusively** for orthotic fittings of the lower limbs and **exclusively** for contact with intact skin.

## 4 Safety

### 4.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

### 4.2 General safety instructions

#### **WARNING**

##### **Operation of motor vehicles**

Risk of accidents due to restricted body function

- ▶ Observe the applicable legal and insurance regulations for the operation of motor vehicles and have your driving ability examined and certified by an authorised agency.

#### **CAUTION**

##### **Contact with heat, embers or fire**

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

**⚠ CAUTION**

**Excessive strain on load-bearing components**

Injuries due to changes in or loss of functionality

- ▶ Only use the product for the defined area of application.
- ▶ If the product has been exposed to extreme strain (e.g. due to falling), take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

**⚠ CAUTION**

**Mechanical damage to the product**

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ Check the product for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, discontinue use of the product and have it checked by authorised, qualified personnel.

**⚠ CAUTION**

**Use on more than one patient**

Skin irritation, formation of eczema or infections

- ▶ Only use the orthosis on one patient.

**⚠ CAUTION**

**Improper cleaning**

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ Clean the product regularly.
- ▶ **Inform the patient.**

**⚠ CAUTION**

**Improper handling**

Risk of falling due to accidentally opening the lock

- ▶ Prior to handing over the orthosis, ensure that the locks are engaged securely and in parallel.
- ▶ Avoid accidentally opening the locks.
- ▶ Instruct the patient in the correct usage of the system joints.

**⚠ CAUTION**

**Incorrect or excessively tight application**

Risk of local pressure and constriction of blood vessels and nerves due to improper or excessively tight application

- ▶ Ensure that the product is applied properly and fits correctly.

**⚠ CAUTION**

**Improper moulding or application**

Injuries or damage to the orthosis due to overloading of the material and improper fit of the orthosis due to breakage of load-bearing components

- ▶ Do not make any improper changes to the orthosis.
- ▶ Always apply the orthosis according to the information in the instructions.

### **⚠ CAUTION**

#### **Using orthosis without padding**

Risk of local pressure points, superficial injuries or skin irritation due to skin contact with rigid or sharp orthosis components

- ▶ Only use the orthosis with undamaged padding.

### **⚠ CAUTION**

#### **Improper handling**

Risk of falling due to accidentally opening or engaging the lock

- ▶ Avoid accidentally opening or engaging the lock.
- ▶ Instruct the patient in the correct handling of the orthosis joints.

### **NOTICE**

#### **Contact with saltwater or water containing chlorine/soap, fresh water or abrasive substances (e.g. sand)**

Damage and premature product wear

- ▶ Following any contact with the substances identified above, promptly clean the product in accordance with the section "Cleaning".

### **NOTICE**

#### **Contact with oils, salves, lotions or other products that contain oils or acids**

Insufficient stabilisation due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

### **NOTICE**

#### **Exposure of the product to unsuitable environmental conditions**

Damage, brittleness or destruction due to improper handling

- ▶ Avoid storage in condensing ambient humidity.
- ▶ Avoid contact with abrasive substances (e.g. sand, dust).
- ▶ Do not expose the product to temperatures below -10 °C (14 °F) or above +60 °C (140 °F) (e.g. sauna, excessive sunlight, drying on a radiator).

## **5 Use**

### **5.1 Application**

- > To apply the orthosis, the user must be sitting down.
- > The orthosis is in Free Walk mode (see Page 10).
- 1) Slide the foot component of the orthosis into the shoe (see fig. 1).
- 2) Put on the shoe (see fig. 2).
- 3) Slide the orthosis over the leg from the front.
- 4) If required, adjust the width of the quick-release closures. Adjust the hook-and-loop closures to do so.
- 5) Fasten the quick-release closures (see fig. 3).

### **5.2 Removal**

- > To remove the orthosis, the user must be sitting down.
- > The orthosis is in Free Walk mode (see Page 10).
- 1) Open the quick-release closures on the orthosis and pull it forward off the leg.

- 2) Remove the shoe.
- 3) Pull the foot component of the orthosis out of the shoe.

### 5.3 Selecting the function modes

#### Selecting Free Walk mode

- ▶ Slide the 3-phase switch to the upper position (see fig. 4).

#### Selecting permanent locking

- 1) Slide the 3-phase switch to the lower position (see fig. 4).
- 2) Push the orthosis ankle joint towards the body and extend the orthosis knee joint.  
→ The knee joint is permanently locked.

#### Selecting free motion mode

- 1) Slide the 3-phase switch to the upper position (see fig. 4).
- 2) Push the orthosis ankle joint towards the body and extend the orthosis knee joint.
- 3) Flex the orthosis knee joint.
- 4) Slide the 3-phase switch to the lower position (see fig. 5).  
→ The orthosis knee joint can be flexed freely at all times.

## 6 Cleaning

Clean the orthosis regularly:

- 1) Remove the straps from the orthosis.
- 2) Fasten all hook-and-loop closures.
- 3) Hand-wash the straps and pads in warm water at **30 °C (86 °F)** using neutral detergent. Rinse thoroughly.
- 4) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

## 7 Disposal

Dispose of the product in accordance with national regulations.

## 8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 8.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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